


# CuddleBug: Getting Started Out of the Box

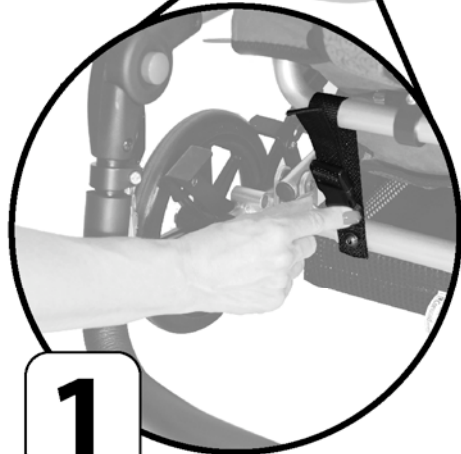

**1**

After removing chair from box, release the **Closure Strap**. The Closure Strap is located at the back of chair, and is attached to the frame between the rear tires.



**2**

To bring the seat back to an upright position, squeeze and hold the **Recline Lever** and lift the seat back upward. The **Recline Lever** is on the right side of the Push Handle.



**3**

Chair is fully upright and in the **Low Position**.



**4**

Apply **Toggle Wheel Locks** by pressing down on lock with foot. Lock is located next to the wheel.



**5**

To raise the chair to the **High Position**, squeeze and hold the High/Low lever while simultaneously pulling in an upward motion. The High/Low lever is located at the center of the seat back.



**6**

Chair is fully upright and in the **High Position**.


**(continues on reverse)**

# Lower Chair to **Low Position**



**1**

Apply **Toggle Wheel Locks** by pressing down on lock with foot. Locks are located next to the wheel.

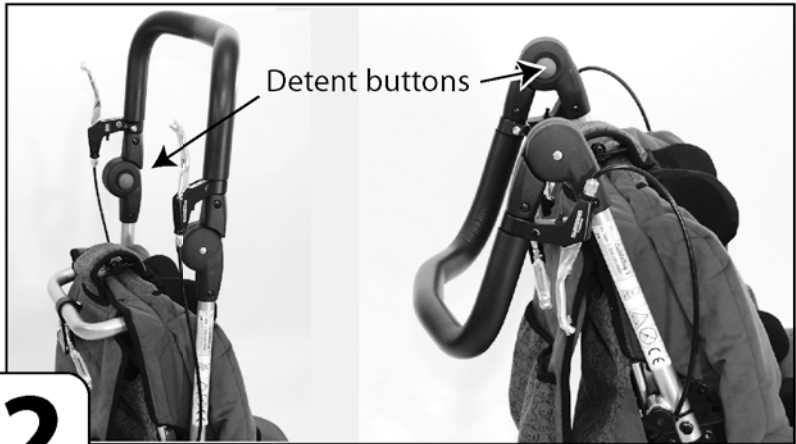

**2**

*Push Handle Up*

*Push Handle Down*

Position the **Push Handle** completely up or down by pressing on the gray detent buttons and adjusting. The gray buttons are located on each side of the push handle.



**3**

To lower the chair to the **Low Position**, squeeze and hold the High/Low lever while simultaneously pushing in a down and forward motion. The High/Low lever is located at the center of the seat back.



**4**

Review the **Quick Start Guide** for directions on the folding, tilting, reclining and High/Low features.

