

Important Information

1

Apply wheel locks and flip up the armrests.



2

Facing the chair, hook one end of grab bar onto chair tubing just above the seat. (NOTE: the hook will not be tight.)



3

Holding **(A)**, the hooked end for stability, pull **(B)**, the opposite end of the grab bar until it is past the side tubing as shown.



Caution: Keep fingers away from moving parts



4



Move hand grip to cover unpainted section.

5



Flip the armrests down.

If you have any questions about how to attach the Grab Bar, please call our Customer Service Department at 888-266-8243 or (310) 618-0111.