

Important Information




Warning:

Do not use tray on chair while being transported in vehicle. Stow the tray in a secure place.

1

Swing the tray mounting brackets outward in direction of arrows (see picture).

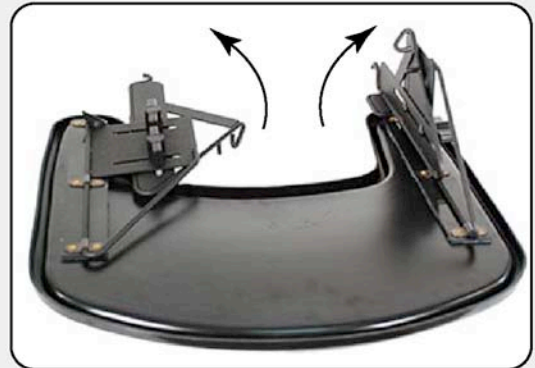

2

Tilt tray slightly (as shown), pull mounting brackets outward and position **(A)** the hooked ends of the tray mounting brackets beneath **(B)** the armrest mounting bolts.




3


Continuing pulling outward on the mounting brackets and attach **(C)** hooked ends onto **(D)** armrest tubes.



Caution: Keep fingers away from moving parts

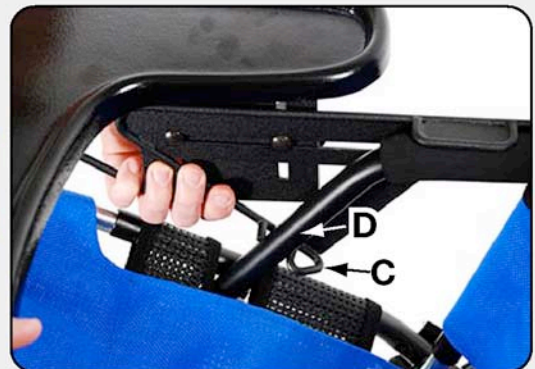

4

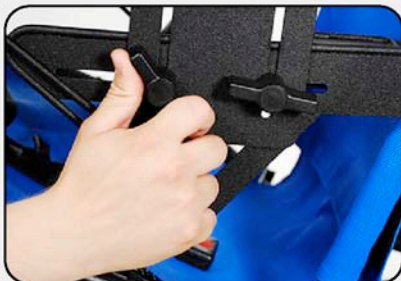

Loosen knobs to adjust tray height, angle or depth.

5



Lift, tilt or pull to desired position. Then tighten knobs to lock tray into place.

If you have any questions about how to attach the Height Adjustable Tray, please call our Customer Service Department at 888-266-8243 or (310) 618-0111.