

Important Information

PARKING BRAKE

Caution:
Improper use could cause premature brake failure, which could damage wheel lock.



STEP 1

To engage parking brake, pull back on red brake lever until locking pin meets solid resistance. Continue applying pressure while rolling wheel slightly forwards and backwards until brake pin fully engages with wheel.



STEP 2


To unlock, push up on Red Brake Lever. Fully disengage wheel lock until you hear "CLICK."

TIRE REMOVAL



STEP 1a REMOVING

To remove, press down and hold the Rear Wheel Locking Pin. Grasp wheel and pull.

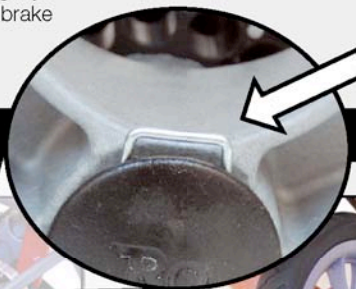

Rear Wheel Locking Pin



STEP 1b

No tools are required to remove tire.

TIRE INSTALLATION



STEP 1a INSTALLING

Caution:

Ensure the wheels are installed properly. You will hear a "click" when wheel locks into place.



STEP 1b

To install, slide wheel onto rear axle and push with ball of hand until it clicks into place.

If you have any questions about the Quick Release Tires, please call our Inside Sales Department at 888-266-8243 or (310) 618-0111.