

## Important Information

### PARKING BRAKE



#### STEP 1

To lock, pull down on Red Brake Lever until it locks into place.



#### STEP 2

To unlock, push up on Red Brake Lever.

### TIRE REMOVAL

Rear Wheel  
Locking Pin



#### STEP 1a REMOVING

To remove, press down and hold the Rear Wheel Locking Pin. Grasp wheel and pull.



#### STEP 1b

### TIRE INSTALLATION



#### STEP 1a INSTALLING

#### Caution:

Ensure the wheels are installed properly. You will hear a "click" when wheel locks into place.



#### STEP 1b

To install, slide wheel onto rear axle and push with ball of hand until it clicks into place.

If you have any questions about the Quick Release Tires, please call our Inside Sales Department at 888-266-8243 or (310) 618-0111.