

Important Information



Squiggles




Caution: Keep fingers free of folding mechanism.

NOTE:
Seating System and Adapter are for use on Convaid Safari Base only.

Unfolding Mobility Base



STEP 1
 Lay base flat onto the floor and release closure strap.

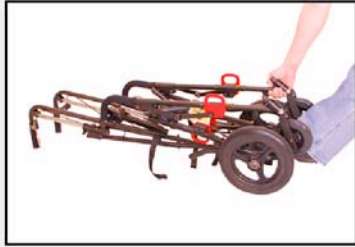


STEP 2a
 Place foot on cross brace and lift frame by 8" - 10" to loosen.



STEP 2b



STEP 3a
 Stand base on it's wheels. Grasp push handle and push down on seat tube.

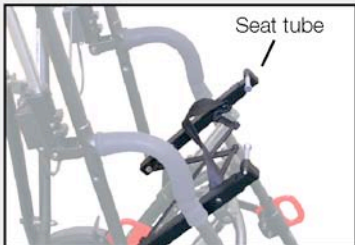

STEP 3b



STEP 4
 Push down on the rear lock brace with your foot, until it is in the locked position.




STEP 8
 Fasten snap to secure closure strap.



STEP 8
 Set toggle wheel locks by pushing down on lock handles.


Attaching Seating Adapter to Base

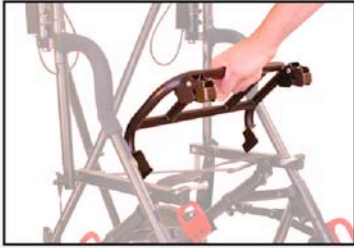


STEP 9a
Insert Seating Adapter by engaging the rear mounting hooks with rear mounting pins and pressing down.



STEP 9b
The rear mounting hooks should be in front of the rear mounting pins, and between the chair tube and the stop washer.


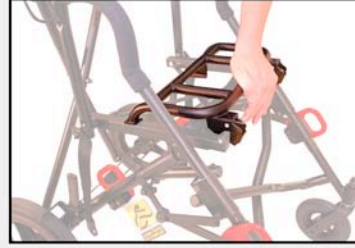


STEP 9c
Push front of seating adapter firmly down until the two clasps lock onto front mounting pins.

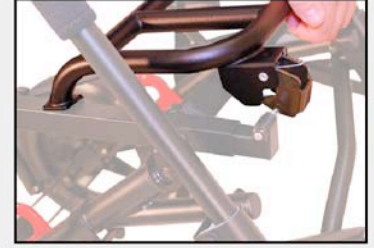


STEP 9d


Attaching Seating System to Seating Adapter

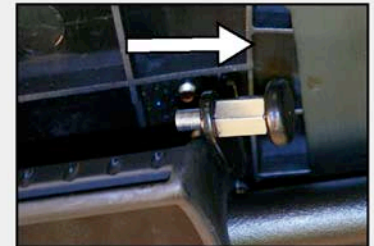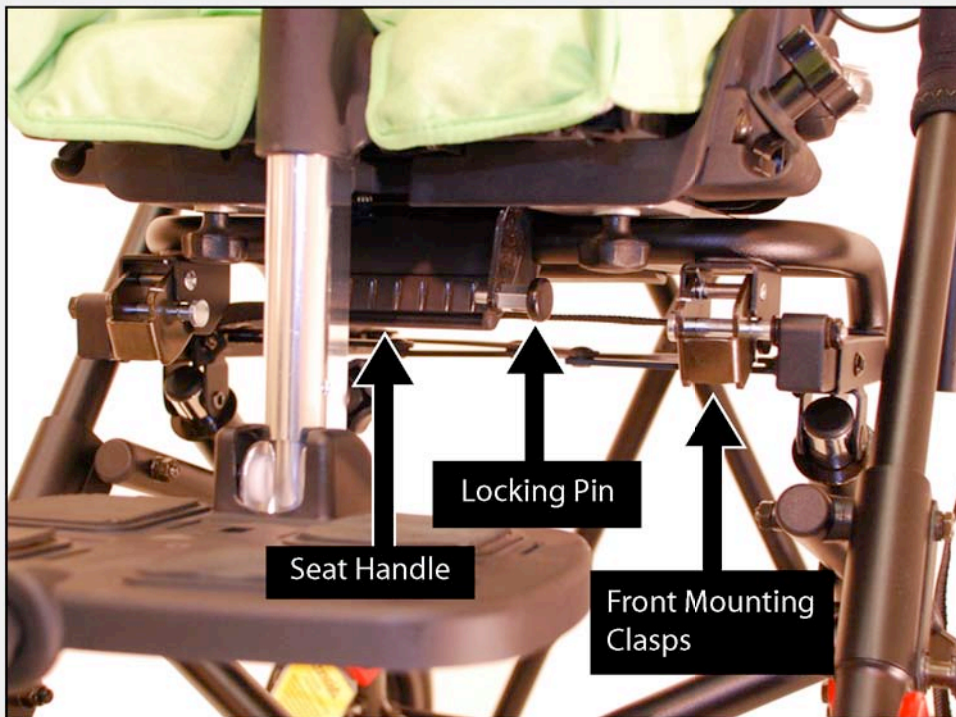


STEP 10a
Release locking pin by pulling in direction of arrow.

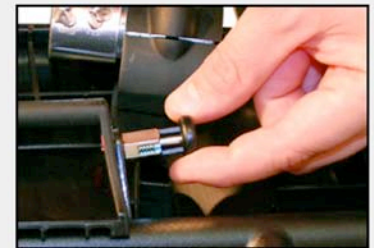

STEP 10b
Turn locking pin 1/4 turn until it remains open.

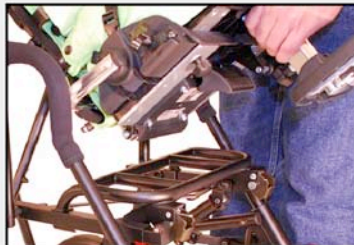

STEP 11
Align rear of seat mounting bracket with seating adapter.

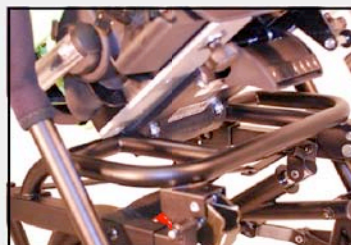

STEP 12A
Insert rear of seat mounting bracket into seating adapter and rotate forward.



STEP 12B




STEP 13

Push firmly down on seating system until the seat handle locks onto front of seating adapter.



STEP 14a


Secure seat handle by turn locking pin 1/4 turn until it closes.

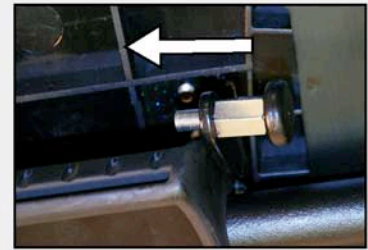

STEP 14b


Folding Instructions

Removing Seating System from Seating Adapter

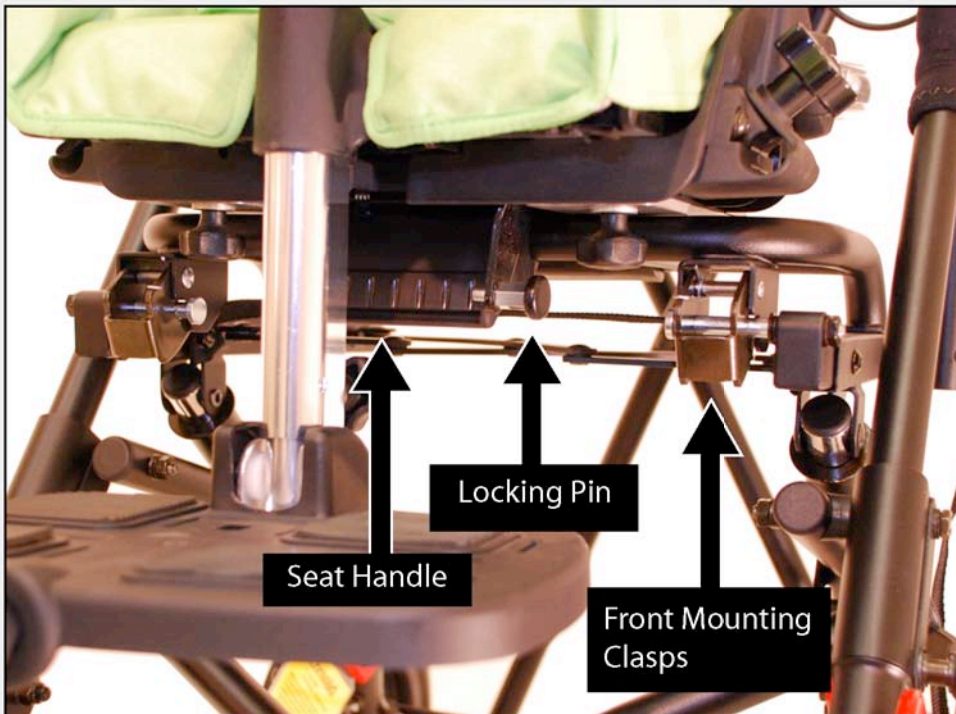

STEP 1

Release toggle wheel lock by pushing up on lock handle.

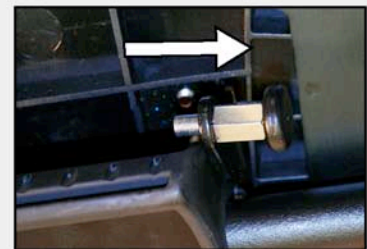


STEP 2a

Unlock locking pin by pulling in direction of arrow and turn locking pin 1/4 turn until it remains open.



STEP 2b

Turn locking pin 1/4 turn until it remains open.

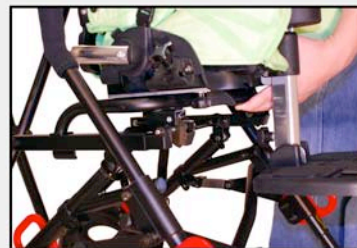


STEP 3a

Remove seating system by pulling seat handle lock upward. Rotate seating system backwards and lift out of mobility base.



STEP 3b

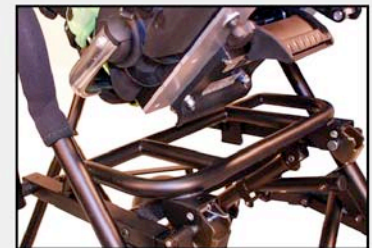

STEP 3c

Removing Seating Adapter from Base



STEP 4a

Release front mounting clasps on seating adapter and lift. At the same time rotate seating adapter backwards and lift out of mobility base.



STEP 4b



STEP 4c



STEP 4d

Folding Mobility Base



TIP: The Mobility Base **MUST** be fully upright and in a locked position before folding.



STEP 5a

While squeezing both tilt levers simultaneously, place foot on rear lock brace and bring the chair to the most upright position (until you hear a "click").



STEP 5b



STEP 6

Unlock rear lock brace by pushing up on center of brace with foot.



STEP 7a

Grasps seat strap and pull.



STEP 7b

Lay mobility base back on push handles and push down on front mount to close frame.



STEP 8

Fasten the closure strap. **Do not use the closure strap to carry.**