

Important Information



Squiggles



Mygo



Caution: Keep fingers free of folding mechanism.


NOTE: Seating System and Adapter are for use on Convaid EZ Base only.

Unfolding Mobility Base



STEP 1

Lay base flat onto the floor and release locking bar.

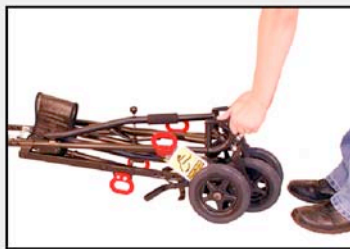

STEP 2

Lift top of frame mount by 2" to loosen frame.



STEP 3

Stand base on it's wheels. Grasp push handle and push down on seat tube.

Attaching Seating Adapter to Base




STEP 4

Set toggle wheel locks by pushing down on lock handles.



STEP 5a

Insert Seating Adapter by engaging the rear mounting hooks with rear mounting pins and pressing down.

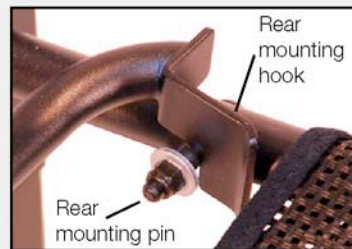

STEP 5b


The rear mounting hooks should be in front of the rear mounting pins, and between the chair tube and the stop washer.




STEP 5c

Push front of seating adapter down forcefully until the two clasps lock onto front mounting pins.

Attaching Seating System to Seating Adapter

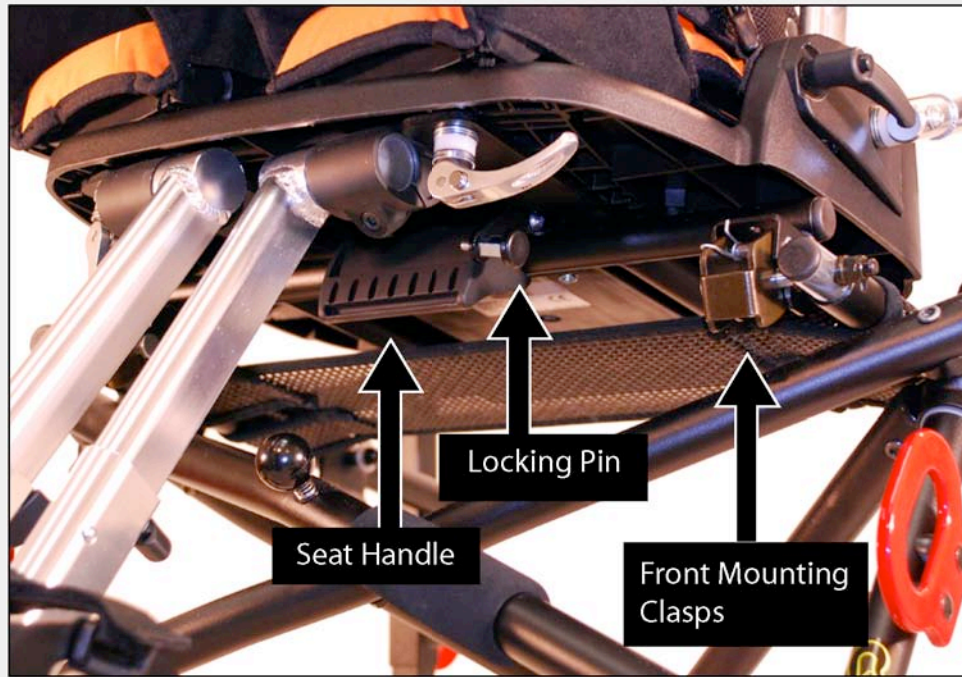

Viewed from front of wheelchair

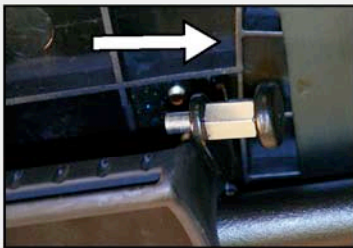

STEP 6a

Release locking pin by pulling in direction of arrow.



STEP 6b

Turn locking pin 1/4 turn until it remains open.




STEP 7

Align rear of seat mounting bracket with seating adapter.



STEP 8a

Insert rear of seat mounting bracket into seating adapter and rotate forward.

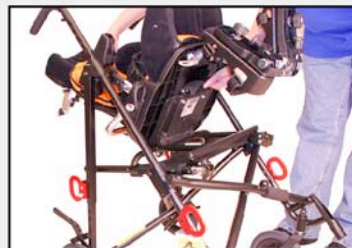


STEP 8b




STEP 9

Push firmly down on seating system until the seat handle locks onto front of seating adapter.

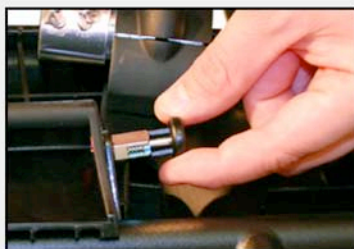


STEP 10a

Secure seat handle by turn locking pin 1/4 turn until it closes.

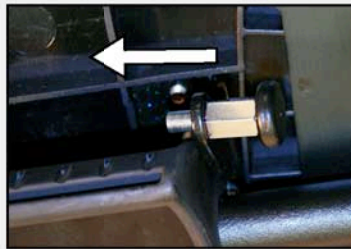

STEP 10b

Folding Instructions

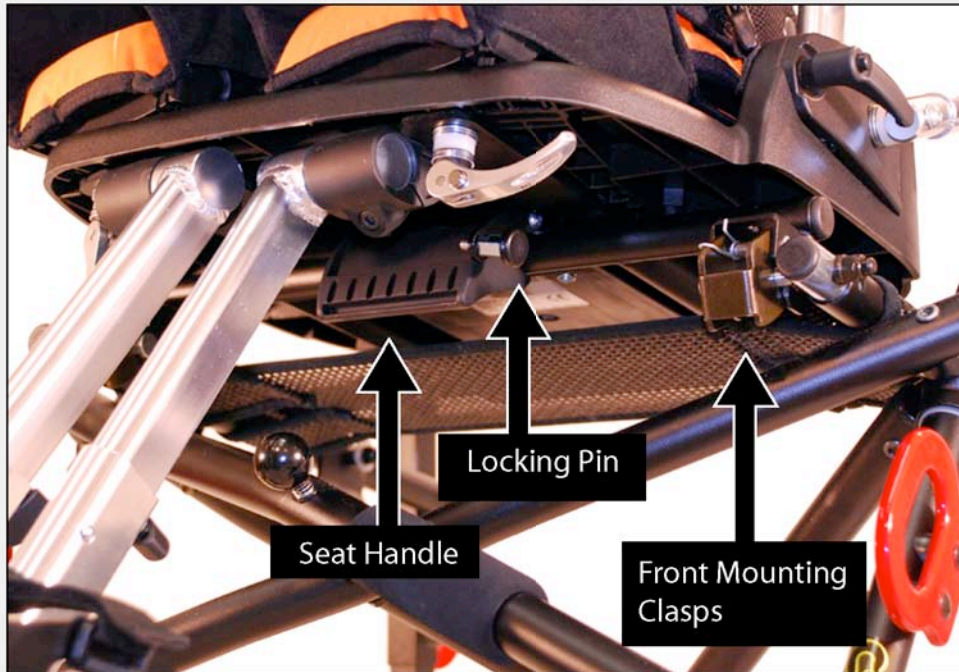


Viewed from front of wheelchair

Removing Seating System from Seating Adapter



STEP 1

Set toggle wheel locks by pushing down on lock handles.

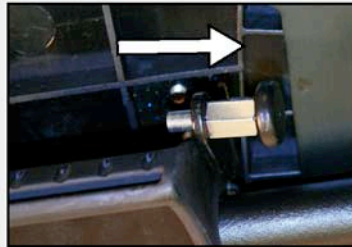

STEP 2a

Unlock locking pin by pulling in direction of arrow and turn locking pin 1/4 turn until it remains open.




STEP 2b




Viewed From Under Seat

STEP 3a


Remove seating system by pulling seat handle lock upward. Rotate seating system backwards and lift out of mobility base.

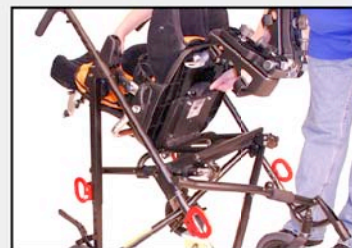

STEP 3b

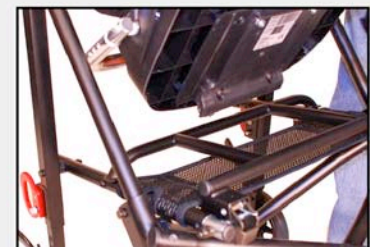


STEP 3c

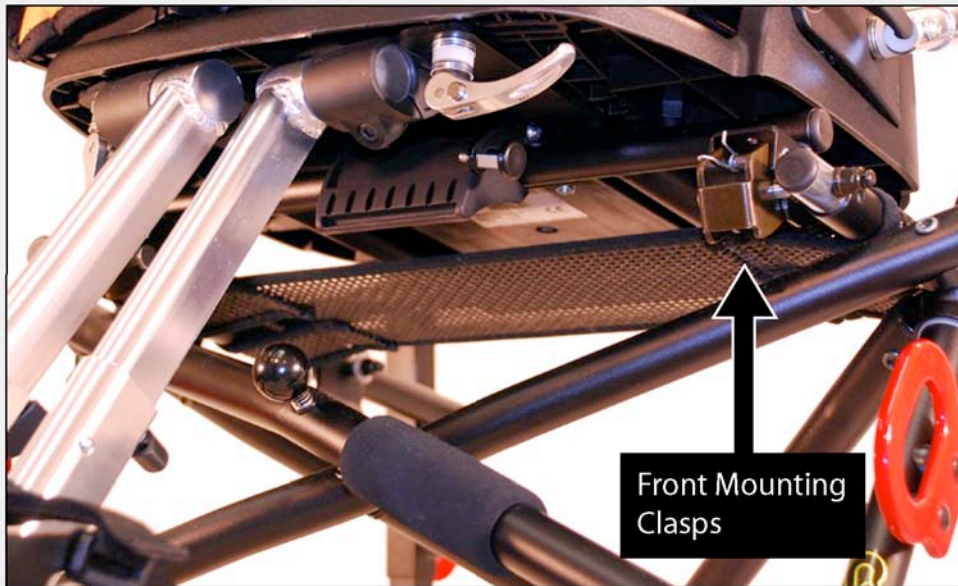

Viewed from front of wheelchair

Removing Seating Adapter from Base

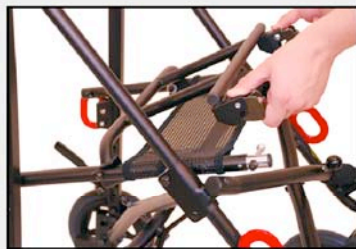


STEP 4a



STEP 4b


Release front mounting clasps on seating adapter and lift. At the same time rotate seating adapter backwards and lift out of mobility base.

Folding Mobility Base

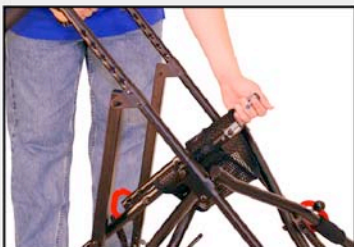


STEP 5

Grasp push handle and pull up on seat tube.

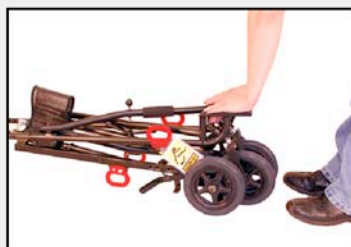

STEP 6

Lay mobility base back on push handles and push down on front mount to close frame.

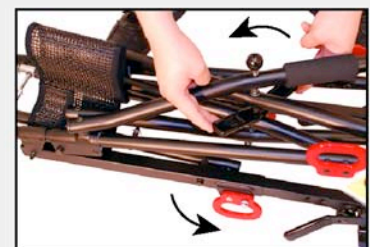

STEP 7

Rotate locking bar until frame is securely locked.