


# Etac Clean


## Reconditioning instructions

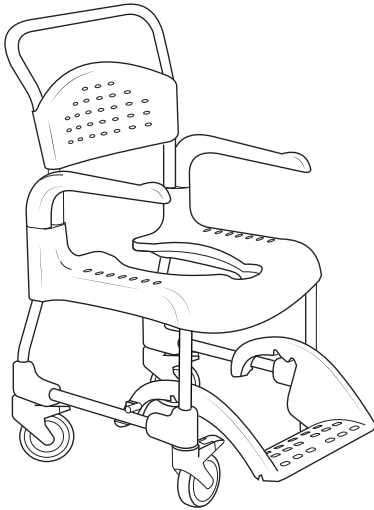


78673B 19-12-11



Clean

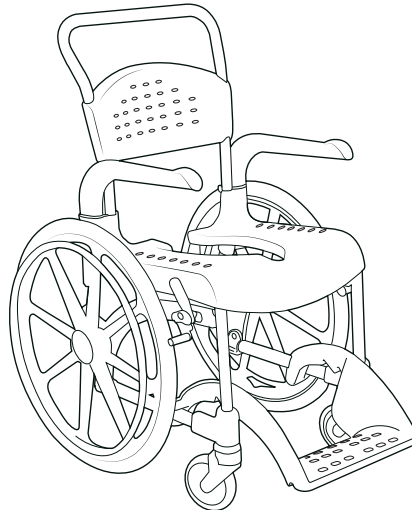

Clean 24"

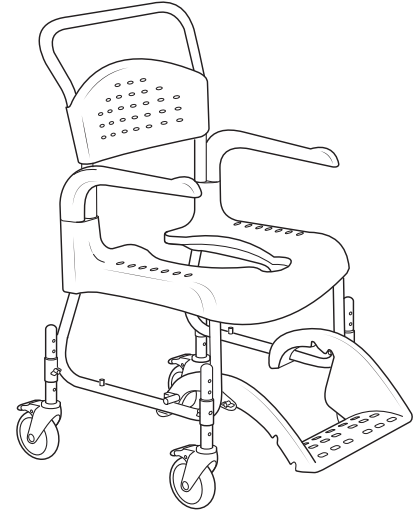

Clean HRB

### Reconditioning/reuse

The frequency of inspection/reconditioning depends on how often and how intensively the product is used. Always use Etac's original parts. For more information about the product, refer to the manual available at [www.etac.com](http://www.etac.com).

Remember that the service life of the product is limited (see manual). This is particularly important when instructing users who are expected to use the product for an extended period of time.

### Cleaning

Washing and cleaning instructions for the product are found on the product as well as in the product's manual. When cleaning the chassis in a washer, the product must be secured in an upright position with locked wheels.

The product can be safely cleaned at 85°C for 3 minutes.

Clean the product with a solvent-free cleaning agent with a pH level between 5 and 9, or with 70% rubbing alcohol.

### Reconditioning

Inspection	Deviation	Solution	Clean	Clean HRB	Clean 24"
Chassis	The chassis has paint damage	Repair the damage with Etac's touch-up paint. RAL 9003 (Gloss 90)	●	●	●
	The chassis has mechanical damage which can affect the product's function/safety	Scrap the product			
Plastic back	The plastic back is damaged, has cracks, burrs or sharp edges	Replace the plastic back			
	The plastic back is not secured	Tighten all screws to 3 Nm torque	●	●	●
	Screw cannot be tightened	Replace plastic back			
Soft back	The soft back is damaged, has tears, holes, broken seams, broken snap locks	Replace the soft back	●	●	●

# Etac Clean

## Reconditioning instructions



78673B 19-12-11

### Reconditioning

Inspection	Deviation	Solution	Clean	Clean HRB	Clean 24"
Arm supports	The arm supports are damaged, have cracks, burrs or sharp edges	Replace the arm supports			
	Arm supports are hard to fit/remove	Clean arm support and mounting			
	Arm supports are hard to fit/remove	Replace malfunctioning part (arm support, seat, arm support lock)	●	●	●
	Arm support cannot be locked	Replace malfunctioning part (arm support, seat, arm support lock)			
Seat	The seat is damaged, has cracks, burrs or sharp edges	Replace the seat			
	The seat is not secured	Tighten all screws under the seat to 3 Nm torque	●	●	●
	Screw cannot be tightened	Replace the seat			
Castor wheel/castor wheel attachment	Castor wheel/castor wheel attachment is damaged, has cracks, burrs or sharp edges	Replace the castor wheel			
	The tread is worn	Replace the castor wheel			
	Castor wheel is stiff/does not rotate	Clean castor wheel			
	Castor wheel is stiff/does not rotate	Replace the castor wheel	●	●	●
	The brake does not lock	Replace the castor wheel			
	The castor wheel is not secured	Tighten all screws beneath the wheel attachment to 4.5 Nm torque			
	Screw cannot be tightened	Replace the castor wheel			
Foot support	The foot support is damaged, has cracks, burrs or sharp edges	Replace the foot support	●	●	●
Height adjustment	Wing nuts are loose	Tighten the wing nuts		●	
	Wing nuts cannot be tightened	Replace screw and wing nut			
Rear wheel adapter	Rear wheel adapter is damaged, has cracks, burrs or sharp edges	Replace the rear wheel adapter			●
	The rear wheel adapter is not secured	Tighten the screws			
Rear wheel	Rear wheel is damaged, has cracks, burrs or sharp edges	Replace rear wheel			
	The tread is worn	Replace rear wheel			●
	Rear wheel is stiff	Replace rear wheel			
	Rear wheel is not secured	Tighten screw/nut			
Brake	The brake is damaged, has cracks, burrs or sharp edges	Replace the brake			●
	The brake does not lock	Adjust the brake position			
	The brake does not lock	Replace the brake			