

**Manatee**

User information appropriate for use by people with visual, reading or cognitive disability      The latest version of all instructions are always available on the R82 website [support.R82.org](http://support.R82.org) and can be printed in larger sizes

Information available in pictogram      The instructions are made as pictograms

Intended use and intended environment including standard options      The Manatee is a bathing chair suitable for disabled children. The chair is designed to place the user in exactly the right sitting position. The depth of the seat can be adjusted, allowing the chair to grow with the child. The product is for indoor use in a bathroom environment. The target group for this device is disabled children with cerebral palsy (CP). Also, children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device. The product is suitable for children with GMFCS level 3-5.

Maintenance instructions      Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
After every use	<ul style="list-style-type: none"> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> <li>- When the chair has been used in water containing chlorine it must be washed out with clean water and dried with a cloth. Make sure that all water is out of the frame</li> </ul>
Daily	<ul style="list-style-type: none"> <li>- Disinfect the product. please check the disinfection section in this User Guide for more information</li> <li>- Clean the anti-slip rubber covers, using a cloth with warm water and mild detergent and wipe them dry</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Ensure that all fasteners are in place and fitted correctly</li> <li>- Be aware of any MAX mark indications before adjustment</li> </ul>
Monthly	<ul style="list-style-type: none"> <li>- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> </ul>
Every 6 months	<ul style="list-style-type: none"> <li>- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty</li> </ul>

Cleaning description      Washing - The cover can be washed at max. 50°C using a mild detergent in a washing machine.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

## Manatee

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Before washing  
Remove the following items and clean them separately: Cover, Cushioning & Accessories

Disinfection description	The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a> (Incl. max load)
Mass	N/A
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings
List of accessories, detachable parts and materials	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	N/A
Transport instructions (unoccupied)	Land and/or air
Sound power level	N/A
Required width of corridor	1500 mm (59")
Minimum turning diameter	3000 mm (118")
Forces on the wall	N/A
Electromagnetic environment warning	N/A
Electromagnetic influence warning	N/A
Life-time	The lifetime of the product in normal use is 5 years