

| Orca | | | | | | | | | | | | | |
|---|--|-----------|-------------|---------------------|---|------------------|---|-----------------|---|---------|--|----------|--|
| User information appropriate for use by people with visual, reading or cognitive disability | The latest version of all instructions is always available on the R82 website support.R82.org and can be printed in larger sizes | | | | | | | | | | | | |
| Information available in pictogram | The instructions are made as pictograms | | | | | | | | | | | | |
| Intended use and intended environment including standard options | The Orca is a bath tub designed to be used in a bathroom. It can be used with or without a bathing seat and is suitable for children up to 50 kg (110 lb). For children needing support whilst sitting/bathing, we recommend the Penguin or the Flamingo seat size 1+2 from R82. The bath tub is mounted on a height adjustable frame to provide the carer with optimum working conditions. The Orca comes in one size and is only for indoor use but can in an emergency be taken outside. | | | | | | | | | | | | |
| Maintenance instructions | <p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1st day</td> <td> <ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Remove and safely dispose the protective film from the bath tub </td> </tr> <tr> <td>Before every use</td> <td> <ul style="list-style-type: none"> - Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts - Check that all castors are moving freely and that all wheel locks/brakes are easily activated </td> </tr> <tr> <td>After every use</td> <td> <ul style="list-style-type: none"> - Disinfect the product. please check the disinfection section in this User Guide for more information - Make sure to empty the bath tub and drain pipe completely </td> </tr> <tr> <td>Monthly</td> <td> <ul style="list-style-type: none"> - All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures </td> </tr> <tr> <td>Annually</td> <td> <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty </td> </tr> </tbody> </table> | Frequency | Description | 1 st day | <ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Remove and safely dispose the protective film from the bath tub | Before every use | <ul style="list-style-type: none"> - Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts - Check that all castors are moving freely and that all wheel locks/brakes are easily activated | After every use | <ul style="list-style-type: none"> - Disinfect the product. please check the disinfection section in this User Guide for more information - Make sure to empty the bath tub and drain pipe completely | Monthly | <ul style="list-style-type: none"> - All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures | Annually | <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty |
| Frequency | Description | | | | | | | | | | | | |
| 1 st day | <ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Remove and safely dispose the protective film from the bath tub | | | | | | | | | | | | |
| Before every use | <ul style="list-style-type: none"> - Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts - Check that all castors are moving freely and that all wheel locks/brakes are easily activated | | | | | | | | | | | | |
| After every use | <ul style="list-style-type: none"> - Disinfect the product. please check the disinfection section in this User Guide for more information - Make sure to empty the bath tub and drain pipe completely | | | | | | | | | | | | |
| Monthly | <ul style="list-style-type: none"> - All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures | | | | | | | | | | | | |
| Annually | <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty | | | | | | | | | | | | |
| Cleaning description | <p>This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.</p> <p>This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.</p> | | | | | | | | | | | | |
| Disinfection description | The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection. | | | | | | | | | | | | |
| Dimensions (Incl. max load) | http://www.R82.org - Select product or click here | | | | | | | | | | | | |
| Mass | N/A | | | | | | | | | | | | |

| Orca | |
|---|--|
| Used in combination with other products | N/A |
| Warning about dangerous combinations | The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. |
| List of accessories, detachable parts and materials | http://www.R82.org - Select product or click here |
| Programmable controller information | N/A |
| Operator control adjustments | Manual |
| Folding and dismantling | N/A |
| Transport instructions (unoccupied) | Land and/or air |
| Sound power level | N/A |
| Turning diameter | NA |
| Forces on the wall | N/A |
| Electromagnetic environment warning | N/A |
| Electromagnetic influence warning | N/A |
| Life-time | The lifetime of the product in normal use is 5 years |