

Cricket

User information appropriate for use by people with visual, reading or cognitive disability The latest version of all instructions are always available on the R82 website support.R82.org and can be printed in larger sizes

Information available in pictogram The instructions are made as pictograms

Intended use and intended environment The Cricket is a foldable buggy which comes in 2 sizes. The seat depth can easily be changed, so the buggy can grow with the child. The accessory program make it possible to give the child extra support if needed. It is low weight and extremely compact when folded, which makes it a joy to bring it anywhere.

The Cricket is suitable for users needing a simple foldable buggy, e.g. children with CP, GMFCS level 2-4.

If mounted with labeled transport fittings the Cricket can be used for forward facing transport in motor vehicles with approved 4 point strap-type tiedown system and approved 3 point belt. For transportation the Cricket is suitable for children in the weight range of 18 kg to specified max user weight for transportation.

The Cricket comes in 2 sizes.

Maintenance instructions Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1st Day	<ul style="list-style-type: none"> - Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference - Store the enclosed tools on/at the product
Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any max mark indications before adjustment
Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
Monthly	<ul style="list-style-type: none"> - All nuts and bolt on this product must be

Cricket	
	<p>checked and tightened to avoid unnecessary failures</p> <ul style="list-style-type: none"> - Oil the swing-away parts. We recommend you use a professional lubricant system, e.g. Veidec Dry Lube <p>Annually</p> <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
Cleaning description	<p>The product can be washed at 60° using a mild detergent for 15-20 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Remove all accessories before washing the main product. Equipment components and utensils shall be scrapped or rough cleaned to remove food particle accumulation. Clean the accessories separately. Please allow the product to dry before reuse.</p> <p>The cushion can be washed at 60° using a mild detergent in a household washing machine</p>
Disinfection description	N/A
Dimensions	http://www.R82.org - Select product or click here
Mass	http://www.R82.org - Select product or click here
Used in combination with other products	N/A
Warning about dangerous combinations	<p>The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.</p> <p>The Cricket is tested for resistance to ignition at level non-ignition in accordance with EN1021-1 and EN 1021-2. The Cricket might lose its flame resistant characteristics if using other than original R82 cushions.</p>
List of accessories, detachable parts and materials	http://www.R82.org - Select product or click here
Programmable controller information	N/A
Operator control adjustments	N/A
Folding and dismantling	http://www.R82.org - Select product or click here
Transport instructions	<p>The Cricket complies with ISO 7176-19, for safe transportation in motor vehicles. The Cricket is approved for transportation of 18-43 kg user weight in vehicles (dependent on size).</p> <p>The user should transfer to the vehicle seat and use the vehicle-</p>

Cricket

installed restraint system whenever possible.

The product must be placed in a forward-facing position, when used as a seat in a motor vehicle. Use an approved 4-point strap-type tiedown system according to ISO 10542-2.

Use an approved 3-point belt according to ISO 10542-1 in the wheelchair. Restraints should not be held away from the body by wheelchair components such as armrests or wheels. It is not enough ONLY to use a hip belt.

All accessories must be removed from the wheelchair and secured separately in the vehicle. Accessories which can not be removed, must be secured to the wheelchair but positioned away from the user with energy-absorbing padding placed between the accessories and the user.

The product should be inspected by the dealer before reuse following involvement in any type of vehicle collision.

The instructions on how to prepare the seat and chair, must be carried through before transportation. Transport fittings must be mounted and labelled as below before use of the Cricket for transportation in motor vehicles.



Sound power level	N/A
-------------------	-----