


Tripp

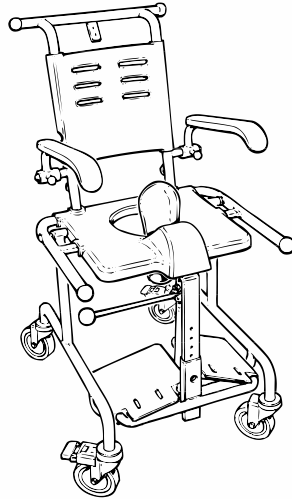

Spare parts can be ordered as per the separate spare parts list for Tripp. The washing and cleaning instructions can be found on the product or in the manual. Remember that the service life of the product is limited (see manual). This is particularly important when instructing users who are expected to use the product for an extended period of time. Check and adjust the following:

The product must be cleaned using conventional non-abrasive cleaning agents. It must be cleaned regularly. The product can be cleaned in a machine at 85° for a maximum of 3 minutes.

- 1) **That the screws on the castors are screwed tightly (33 Nm) and that they are locked with a thread-locking fluid.**
If the castors are loose or have to be changed due to wear:
 - Fully unscrew the bolt that attaches to the chassis beam, and clean the thread using a cloth, if necessary (the castor must first be removed from the fork).
 - Drip the thread-locking fluid (equivalent to Loctite 243 or equivalent) on the screw thread according to the manufacturer's instructions and tighten the bolt (torque wrench: 33Nm).

Tools: Two 13 mm socket wrenches, one 8 mm Allen key

- 2) **That the castors work as intended.**
 - That the locking function works.
 - That the castors lock satisfactorily. The castors must not roll in locked position.
 - That the chair rolls satisfactorily.
- 3) **That armrest mountings and armrests are intact and that the armrests have been properly mounted.**
 - Apply pressure to the armrests using your hands in various ways to ensure that the armrests and mountings are intact.
 - Make sure the locking function is working satisfactorily.

Tools: 6 mm Allen key

- 4) **That the footrests have been properly mounted and that they work satisfactorily.**
 - The footrests must be able to be folded up.
 - The footrests must be height-adjustable.

Tools: 5 mm Allen key

- 5) **That detachable items are in good condition and are Etac's original products.**
 - That the seat is intact.
 - That the back pad is intact.

- 6) **That the pipe end terminals of the product are fitted with plastic plugs.**
 - Use the product sheet to make sure that all end terminals are in place.

- 7) **That the gas piston is properly mounted and that it is operational in all positions.**
 - That the chair does not by itself change inclination in any position:
 - Make sure that the chair and the seat are level, that it tilts backwards at approx. 15°, and that it can tilt fully backwards.

Tools: 5 mm Allen key, 19 mm socket wrench.

1A15510.1

18-08-06

Etac AB, Box 203, SE-334 24 Anderstorp

T: +46 (0) 371-58 73 00, F: +46/371-58 73 90

www.etac.se