

**Strong Base**

<p>User information appropriate for use by people with visual, reading or cognitive disability</p>	<p>The latest version of all instructions are always available on the R82 website <a href="http://www.etac.com/">http://www.etac.com/</a> and can be printed in larger sizes</p>
<p>Information available in pictogram</p>	<p>The instructions are made as pictograms</p>
<p>Intended use and intended environment including standard options</p>	<p><b>Intended purpose</b> The device is a frame for seating systems for children and adults with disabilities.</p> <p><b>Indications</b> The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.</p> <p>The user is the person sitting in the seat used with the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.</p> <p><b>Contra indications</b> There are no known contra indications</p> <p><b>Precautions</b> There are no known precautions</p> <p><b>Application areas</b> The product is for indoor use</p>

<p>Maintenance instructions</p>	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1"> <thead> <tr> <th data-bbox="574 1321 718 1355">Frequency</th> <th data-bbox="718 1321 1372 1355">Description</th> </tr> </thead> <tbody> <tr> <td data-bbox="574 1355 718 1444">1<sup>st</sup> Day</td> <td data-bbox="718 1355 1372 1444"> <ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store them in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul> </td> </tr> <tr> <td data-bbox="574 1444 718 1724">Daily</td> <td data-bbox="718 1444 1372 1724"> <ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fitted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any max mark indications before adjustment</li> </ul> </td> </tr> <tr> <td data-bbox="574 1724 718 1881">Weekly</td> <td data-bbox="718 1724 1372 1881"> <ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> </ul> </td> </tr> <tr> <td data-bbox="574 1881 718 2049">Monthly</td> <td data-bbox="718 1881 1372 2049"> <ul style="list-style-type: none"> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> <li>-</li> </ul> </td> </tr> </tbody> </table>	Frequency	Description	1 <sup>st</sup> Day	<ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store them in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul>	Daily	<ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fitted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any max mark indications before adjustment</li> </ul>	Weekly	<ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> </ul>	Monthly	<ul style="list-style-type: none"> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> <li>-</li> </ul>
Frequency	Description										
1 <sup>st</sup> Day	<ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store them in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul>										
Daily	<ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fitted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any max mark indications before adjustment</li> </ul>										
Weekly	<ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> </ul>										
Monthly	<ul style="list-style-type: none"> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> <li>-</li> </ul>										

<b>Strong Base</b>	
	Annually - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
Cleaning description	<p><b>Warewashing machine</b> This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.</p>
Disinfection description	The product can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the upholstery, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	<a href="http://www.etac.com/">http://www.etac.com/</a> - Select product or click <a href="#">here</a>
Mass	<a href="http://www.etac.com/">http://www.etac.com/</a> - Select product or click <a href="#">here</a>
Maximum slope	<a href="http://www.etac.com/">http://www.etac.com/</a> - Select product or click <a href="#">here</a>
Type(s) of tyres	Solid castors
Used in combination with other products	<p>Can be used in combination with</p> <ul style="list-style-type: none"> <li>• Panda Futura 5 seat</li> <li>• x:panda seats 4S, 4M, 4L</li> </ul> <p>General combination scheme available at <a href="http://www.etac.com/">http://www.etac.com/</a> go to support documents and- Select Chart transport overview or click <a href="#">here</a></p>
Warning about dangerous combinations	<p>When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled. Applicable requirements examples:</p> <ul style="list-style-type: none"> <li>• Alignment of user weight/mass of the combined seat and base</li> <li>• Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety</li> </ul>
List of accessories, detachable parts and materials	<a href="http://www.etac.com/">http://www.etac.com/</a> - Select product and go to accessories section
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	N/A
Used in motor vehicles	N/A
Transport instructions	N/A
Sound power level	N/A