

**Rabbit**

User information appropriate for use by people with visual, reading or cognitive disability      The latest version of all instructions are always available on the R82 website [support.R82.org](http://support.R82.org) and can be printed in larger sizes

Information available in pictogram      The instructions are made as pictograms

Intended use and intended environment      The Rabbit is a mobile standing frame providing users the possibility of moving around and orientate oneself. The standing position enables the child to be an active participant, whilst stimulating senses and giving the child more independence.

The Rabbit is suitable for many users e.g. children with CP, GMFCS level 2-5.

The Rabbit comes in 4 sizes.

Maintenance instructions      Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1 <sup>st</sup> Day	<ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store it in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul>
Daily	<ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fitted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any MAX mark indications before adjustment</li> </ul>
Weekly	<ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> <li>- Check that the angle mechanism functions correctly and that the safety lock is working correctly</li> </ul>
Monthly	<ul style="list-style-type: none"> <li>- Disinfect the product.</li> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> </ul>

Rabbit	
	<ul style="list-style-type: none"> <li>- The product has to be checked and tightened by a qualified technician at least every 12 months to avoid failures.</li> </ul>
Annually	<ul style="list-style-type: none"> <li>- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be sub-standard or faulty</li> </ul>
Cleaning description	This product can be washed at 60°C using a mild detergent in 15-20 minutes in a washing machine de-signed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.
Disinfection description	The product can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Mass	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Programmable controller information	N/A
Operator control adjustments	N/A
Folding and dismantling	N/A
Transport instructions	N/A
Sound power level	N/A