

Now You're Going Places

Chairs • Carrot 3

How to Attach the 3D Body Balance Pads

1. Attach the 3D body balance pads with Velcro® and position them where the user needs assistance with balance and position. **(Fig. 1)**



Fig. 1

How to Attach the Pelvis Lock Harness

Seat Depth Adjustment Knob

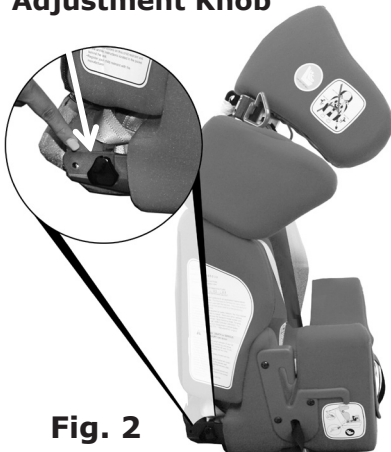


Fig. 2

1. Turn seat depth adjustment knob counter clockwise and remove it. **(Fig. 2)**
2. Locate the left hand side of the pelvis lock harness and insert the left hand side into the metal slot. **(Fig. 3)**
3. Re-insert the seat depth adjustment knob through the hole on the pelvis lock harness. **(Fig. 4 and 5)**
4. Repeat Steps 1-3 for right hand side.



Fig. 3



Fig. 4



Fig. 5



Pelvis Lock Harness

How to Attach the Air Mesh Cushion Set

1. Peel off Velcro® stickers **(Fig. 6)**
2. Place air mesh liner directly on top of the Carrot upholstery. **(Fig. 7)**
3. Velcro® the air mesh support cushions onto the Carrot. **(Fig. 8)**



Fig. 6



Fig. 7



Fig. 8

If you have any questions concerning the Carrot 3, please call your Convaid Representative at 888 CONVAID (266-8243) or +1 310 618-0111.