

## Replace Chest Support

1. Remove standard chest support by unzipping the back to remove cover. **(Fig. 1)**
2. With a razor blade, cut the right and left side of white tape to separate standard chest support from attached buckle. **(Fig. 2)**
3. Remove the large chest support.
4. To attach the small chest support, align the small chest support with the attached car seat buckle. **(Fig. 3)**
5. To secure chest support with car seat buckle – wrap provided white tape around chest support and belt buckle. **(Fig. 4)**
6. Take provided chest support cover for small chest support and wrap it around the chest support until completely covered. **(Fig. 5)**
7. Secure the closure by zipping the back side of the chest support cover. **(Fig. 6)**



**Fig. 1**



**Fig. 2**



**Fig. 3**



**Fig. 4**



**Fig. 5**




**Fig. 6**

### Chest Support Kit



## Carrot 3 Head Support Replacement

1. Detach straps from shoulder harness bracket located on the back. **(Fig. 1)**
2. Pull shoulder straps through the front side of the shoulder harness holder. **(Fig. 2)**
3. Remove the standard head support by simultaneously pushing the two gray buttons **(Fig. 3)** and pull head support up. **(Fig. 4)**
4. Remove screws on each side of the head support with a Phillips screwdriver **(Fig. 5)**
5. Use hole 1,2, or three to attach small head support to desirable position, and reinsert screws. **(Fig. 6)**
6. Reverse steps 1 through 3 to attach small head support.

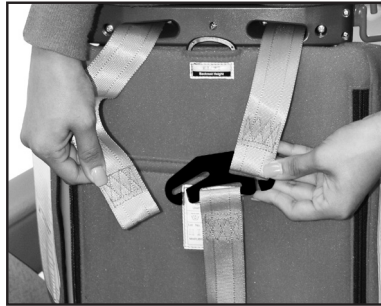

**Fig. 1**



**Fig. 2**




**Fig. 3**



**Fig. 4**



**Fig. 5**

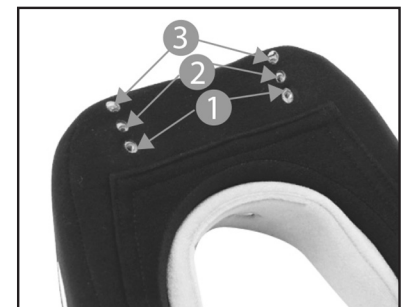

**Fig. 6**