



Step 1

Remove cushion.



Step 2a

Unfasten cushion support upholstery and pull forward.



Step 2b

Using the Tilt Lever slightly tilt the chair to expose the Seat Tube Retaining Screw.



Step 3a

Using tools provide with each wheelchair. Hex key 5/32" and 7/16" remove the Seat Tube Retaining Screw.



Step 3b

Adjust the Seat Depth to desired length. Reinsert Seat Tube Retaining Screw and tighten bolt. Repeat sets 3a and 3b for opposite side..



Step 4

After completing all adjustments repeat step 1 and 2 in reverse order.

If you have any questions concerning the Safari Seat Depth Adjustments, please call our Inside Sales Department at (888) 266-8243 or (310) 618-0111.