


WHEELS

12.5" x 2" Rear Pneumatic Knobby Wheel Assy - Each


994748

Kit Includes

| Item | Description | Qty |
|------|---|-----|
| 1 | 12.5" Knobby Pneu Rear Wheel (Gray Hub) | 1 |

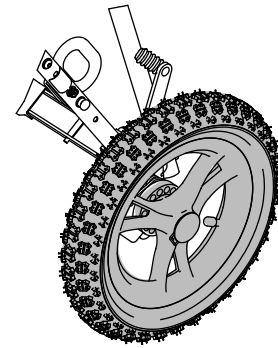

Assembled

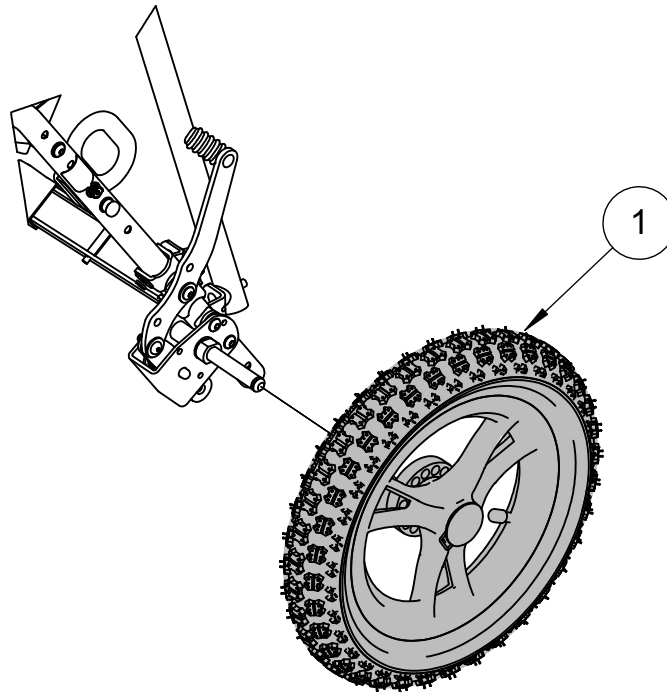

If you have any questions about attaching the Rear Wheel, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

INSTRUCTIONS

12.5" x 2" Rear Pneumatic Knobby Wheel Assy - Each

1. To remove, press down and hold the Rear Wheel Locking Pin. Grasp wheel and pull.
2. No tools are required to remove wheel.

Wheel Installation

Caution: Ensure the wheels are installed properly. You will hear a "click" when wheel locks into place.

3. To install, slide wheel onto rear axle and push with ball of hand until it clicks into place.