


## WHEELS

### 11.5" x 2.5" Rear Solid Wheel Assy - Each


# 994746

#### Kit Includes

Item	Description	Qty
1	Rear Wheel 290mm x 60mm (Rolko)	1

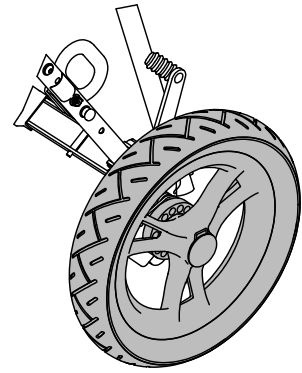

Assembled

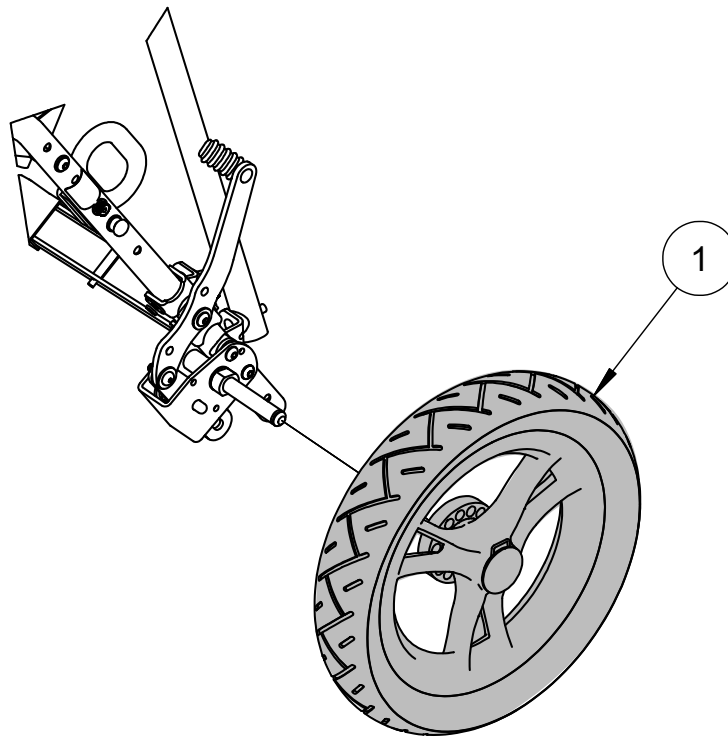

If you have any questions about attaching the Rear Wheel, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

## **INSTRUCTIONS**

---

### **11.5" x 2.5" Rear Solid Wheel Assy - Each**

1. To remove, press down and hold the Rear Wheel Locking Pin. Grasp wheel and pull.
2. No tools are required to remove wheel.

#### **Wheel Installation**

**Caution: Ensure the wheels are installed properly. You will hear a "click" when wheel locks into place.**

3. To install, slide wheel onto rear axle and push with ball of hand until it clicks into place.