


WHEELS

12.5" x 2" Rear Solid Knobby Wheel Assy - Each


994747

Kit Includes

Item	Description	Qty
1	12.5" Knobby Solid Rear Wheel (Gray Hub)	1

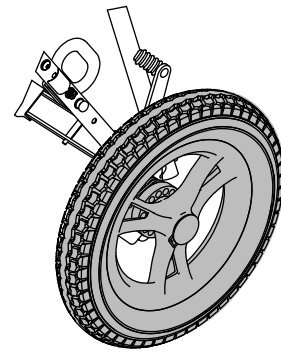

Assembled

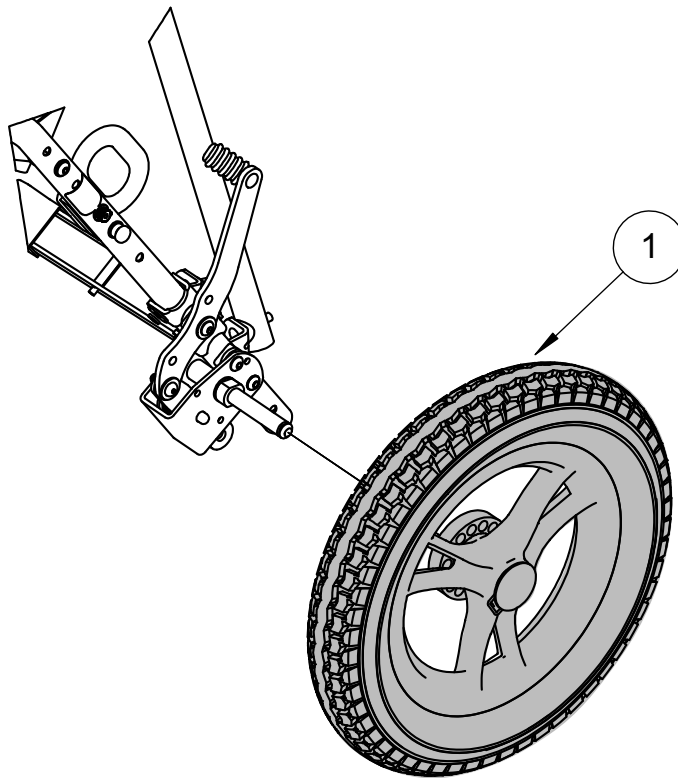

If you have any questions about attaching the Rear Wheel, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

INSTRUCTIONS

12.5" x 2" Rear Solid Knobby Wheel Assy - Each

1. To remove, press down and hold the Rear Wheel Locking Pin. Grasp wheel and pull.
2. No tools are required to remove wheel.

Wheel Installation

Caution: Ensure the wheels are installed properly. You will hear a "click" when wheel locks into place.

3. To install, slide wheel onto rear axle and push with ball of hand until it clicks into place.