


FOOTPLATE

Angle Adjustable Footplate - Left


992621

Kit Includes

Item	Description	Qty
1	Pivot Footplate Assembly-Standard, LH	1

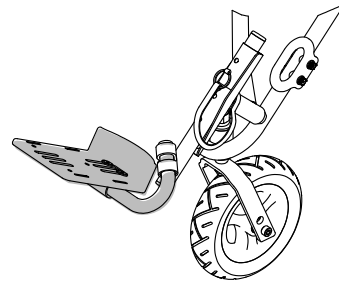

Assembled

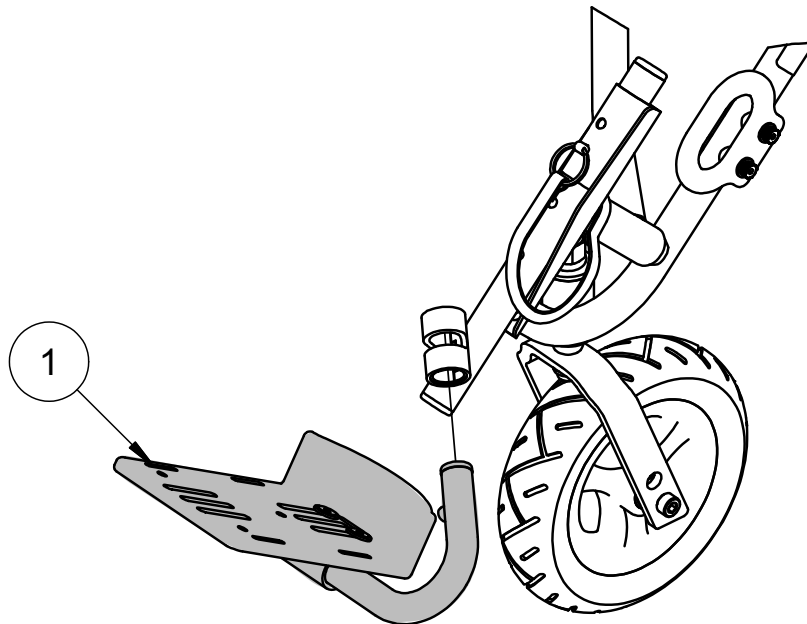

If you have any questions about attaching the Footplate, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

INSTRUCTIONS

Angle Adjustable Footplate - Left

1. Remove the existing footplate assembly by pressing the detent button then pulling the footplate off.
2. Insert the new footplate assembly into the footplate adjustment tube then press the detent button to allow the footplate assembly to slide in and lock in place.