


## UPHOLSTERY

### Heavy Duty Textilene Seat Bottom Upholstery

# 990295

Item	Description	Qty
1	CX-14 Heavy Duty Textilene Seat Bottom Upholstery	1

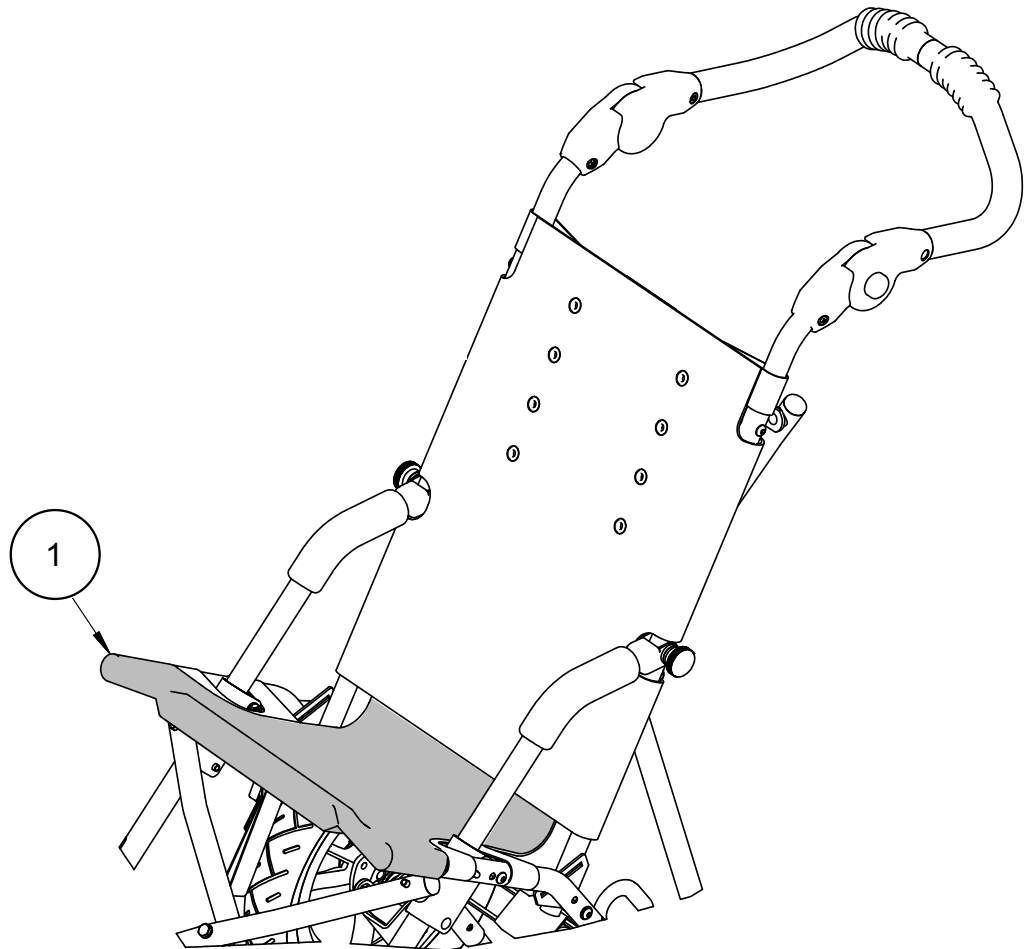

If you have any questions about attaching the Upholstery, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

 **Convaid**  
*Now You're Going Places*

### Fabric Removal Instructions

1. Partially fold the chair to relieve fabric tension (see fold instruction on page 4 in User Guide). The lower seat panel is attached to the back panel with velcro. Remove Velcro® from the top of the seat bottom. Press grey detent pin at the end of the seat extension, and take off upholstery.
2. To remove the top portion of the seat, remove the Velcro® straps that attach the seat Upholstery to the frame, and take off upholstery. Lift fabric off seat tubes and away from upper handles. The seat-reinforcing strap should remain on frame.