


PELVIC POSITIONING

Medical Thigh Support - Abductor

995930

Item	Description	Qty
1	Medical Thigh Support - Abductor	1

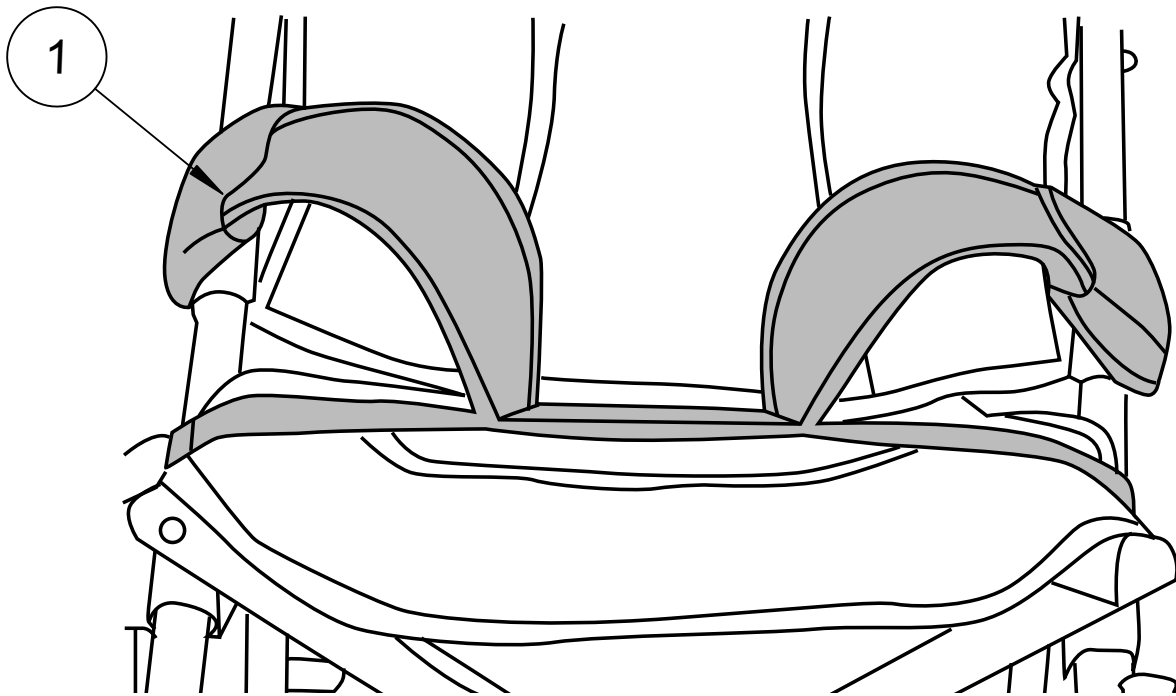


If you have any questions about attaching the Medical Thigh Support, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

 **Convaid**
Now You're Going Places

INSTRUCTIONS

Medical Thigh Support - Abductor

1. Separates the thighs to improve hip alignment and stabilize sitting posture. Degree of abduction can be varied and can favor one side. The abductor flaps wrap over the user's thighs from the inside to the outside.
2. The straps buckle around the seat tube or the armrest. They can also be wrapped around the armrest tube twice for high tone children.
3. The abductor is attached with screws at the end of the seat tubes.

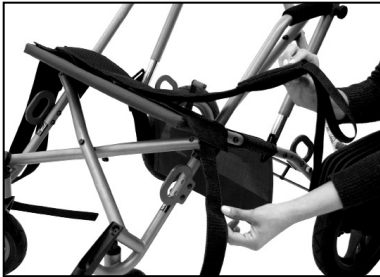

If you have any questions about attaching the Medical Thigh Support, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

 **Convaid**
Now You're Going Places