


**ACCESSORIES**

**Headrest Cover (Canopy)**

**# 995922**

Item	Description	Qty
1	Headrest Cover (Canopy)	1

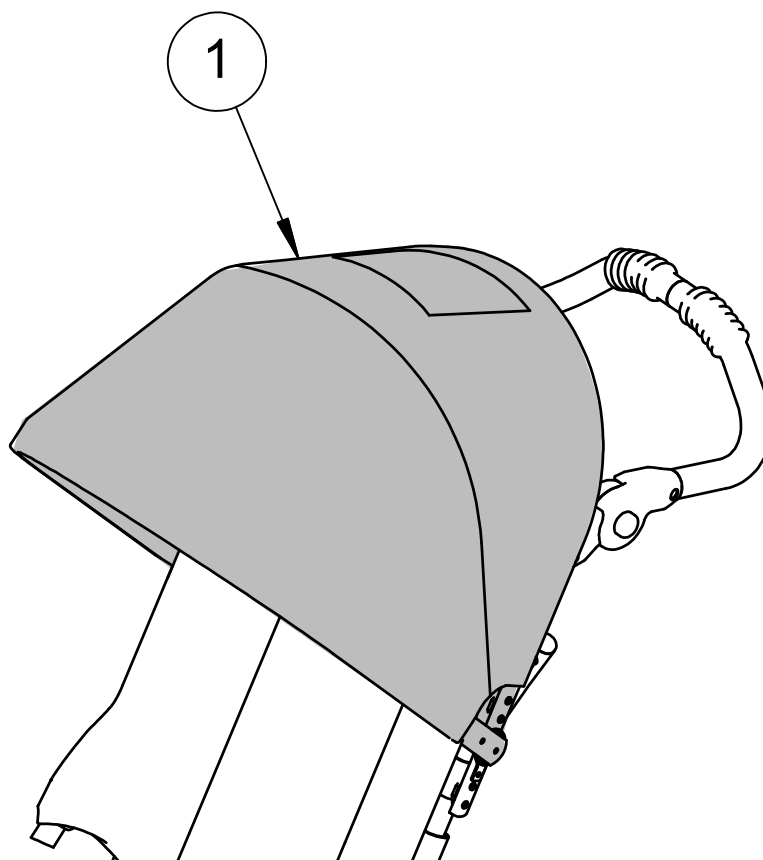

If you have any questions about attaching the Headrest Cover, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

## INSTRUCTIONS

---

### Attach Canopy

1. Remove Canopy from box.
2. Align plastic Canopy Retaining Clips with upper tubing on back of the chair.  
Push on clips with the ball of your hand until they snap into place.

**Caution: Keep fingers away from moving parts.**

3. To open, grasp top of Canopy and push forward and rotate downward.
4. To close the Rear Flap of Canopy, attach the Velcro® strap to Velcro® receiving patch on the edge of the Canopy Hood.
5. To fold, repeat above steps in reverse order.