


**ACCESSORIES**

**Mosquito Net**

**# 996623**

Item	Description	Qty
1	Mosquito Net	1

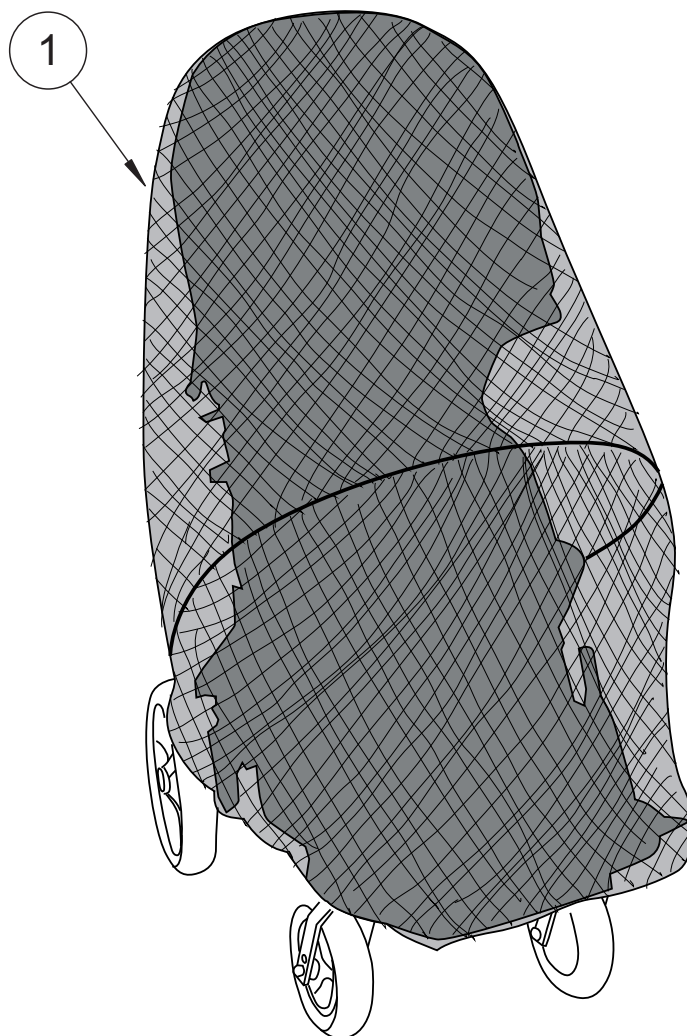

If you have any questions about attaching the Mosquito Net, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

## INSTRUCTIONS

---

### Mosquito Net

1. Take the top of the mosquito net and wrap it around the top of the canopy.  
**You can identify the top of the mosquito net by the two black ties.**
2. Arrange the mosquito net around the wheelchair and pull the bottom of the mosquito net to wrap around the footplate.
3. Find the two Velcro® straps on the right and left side of the mosquito net.
4. Secure mosquito net to wheelchair by wrapping Velcro® straps around each side of wheelchair frame.

**NOTE: For smaller size wheelchairs, the Velcro® straps may connect to each other behind the chair for a tighter fit.**

Available in two sizes:

Small – fits chair sizes 10-14

Large – fits chair sizes 16-18