


HEAD SUPPORT

Padded Headwings

995933

Item	Description	Qty
1	Padded Headwings	1

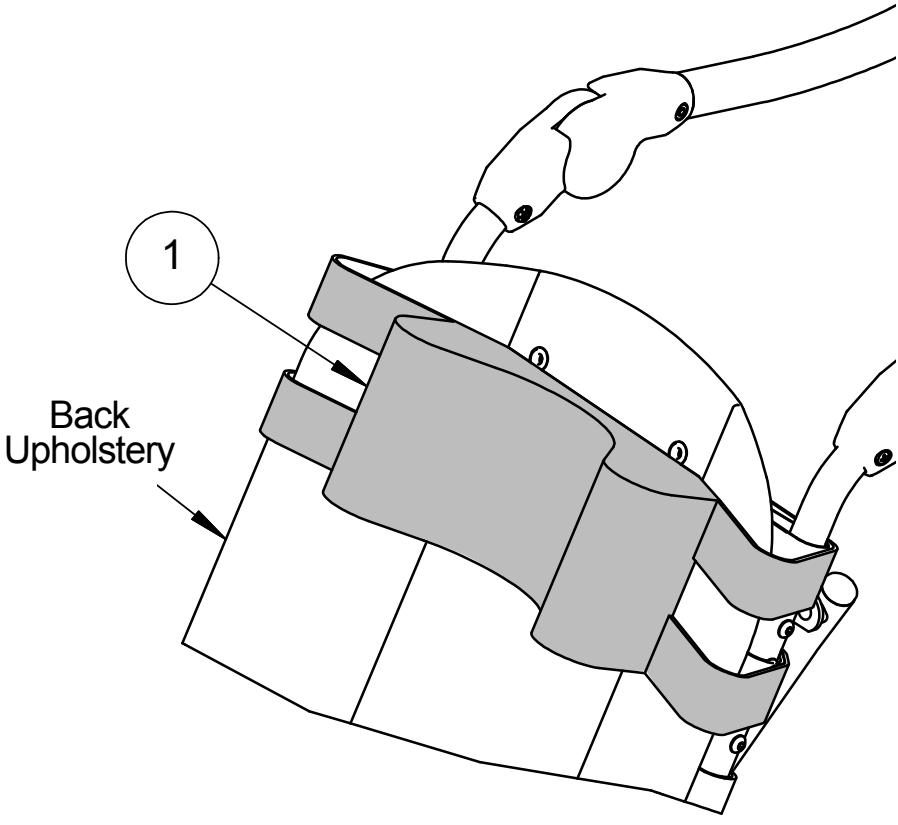

If you have any questions about attaching the Padded Headwings, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

INSTRUCTIONS

Padded Headwings

- Adjustable padded headwings provide soft foam support for midline positioning. The padded headwings can be attached at any height by wrapping the Velcro® straps around the frame and attaching at the back of the chair. Secure headwings by tying laces through the grommet holes in the back of the seat fabric or headrest extension.