


FOOT & LEG POSITIONING

Calf Panel

996615

Item	Description	Qty
1	Calf Panel	1

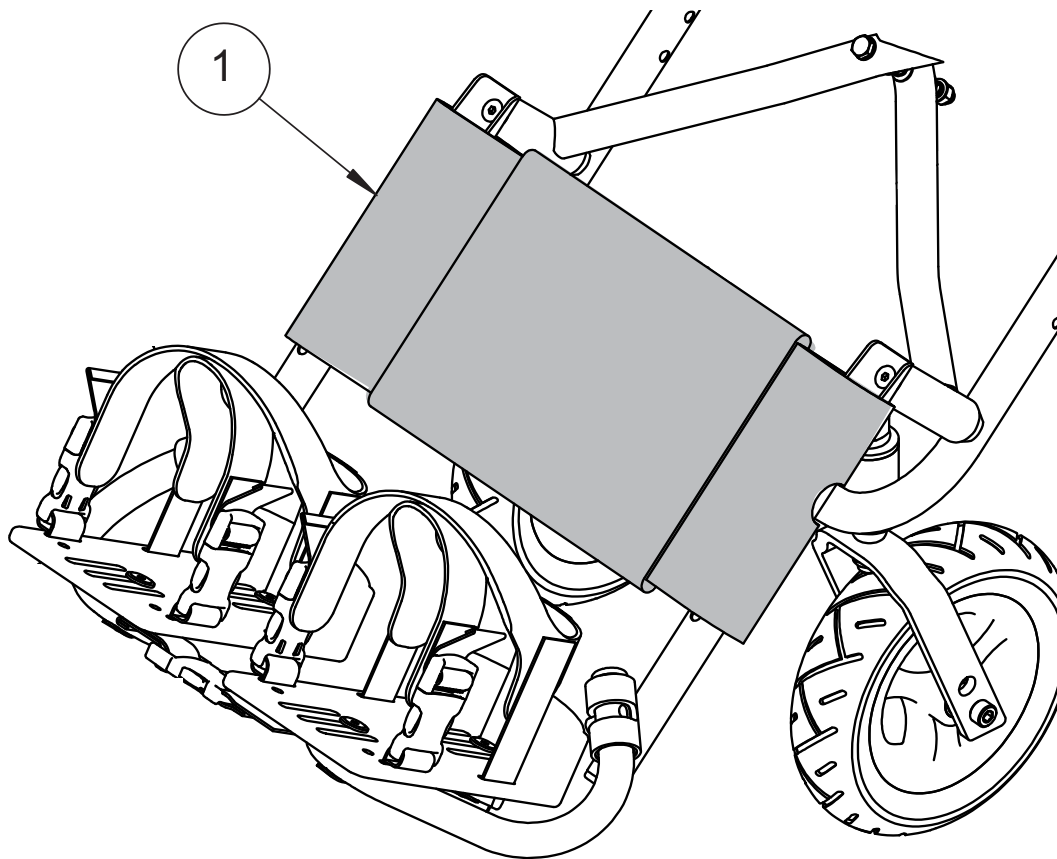

If you have any questions about attaching the Calf Panel, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

INSTRUCTIONS

Calf Panel

1. Lay Calf Panel flat and unfasten the Velcro® straps.
2. Attach Velcro straps around Footrest Extension Tube.
3. Position Calf Panel across chair and attach Velcro® Straps around opposite Footrest Extension Tube.