


ACCESSORIES

Saddle Bags

996620

 Kit Includes

Item	Description	Qty
1	Saddle Bag	2

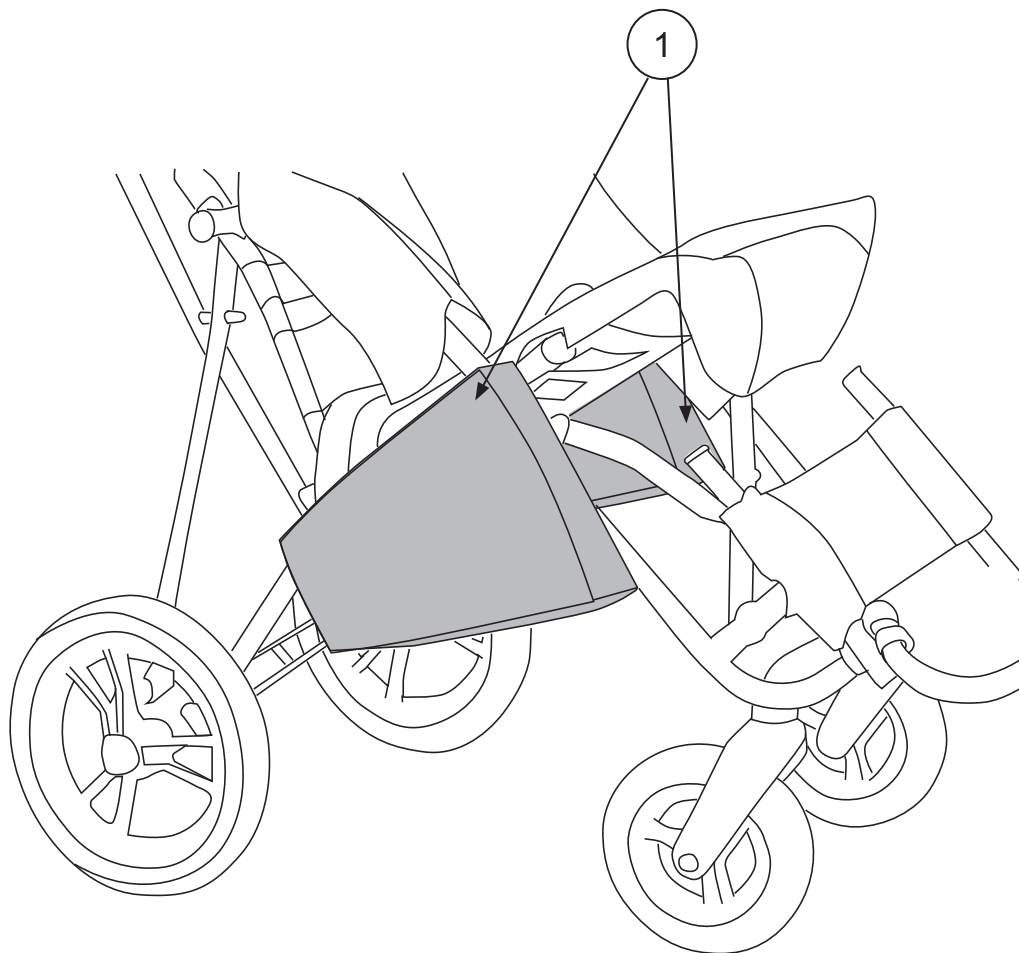

If you have any questions about attaching the Saddle Bags, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

INSTRUCTIONS

Saddle Bags

1. To install, insert Front Button Strap into opening in Mesh Seat Bottom and over Seat Tube.
2. Verify the strap is in front of the Seat Tube Retaining Bolt, then snap button shut.
3. Wrap Rear Button Strap around Seat Tube and snap shut.

If you have any questions about attaching the Saddle Bags, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

 **Convaid**
Now You're Going Places