


TRUNK POSITIONING

Soft Adjustable Lateral Support - Single Flap

995935

Item	Description	Qty
1	Soft Adjustable Lateral Single Flap	1

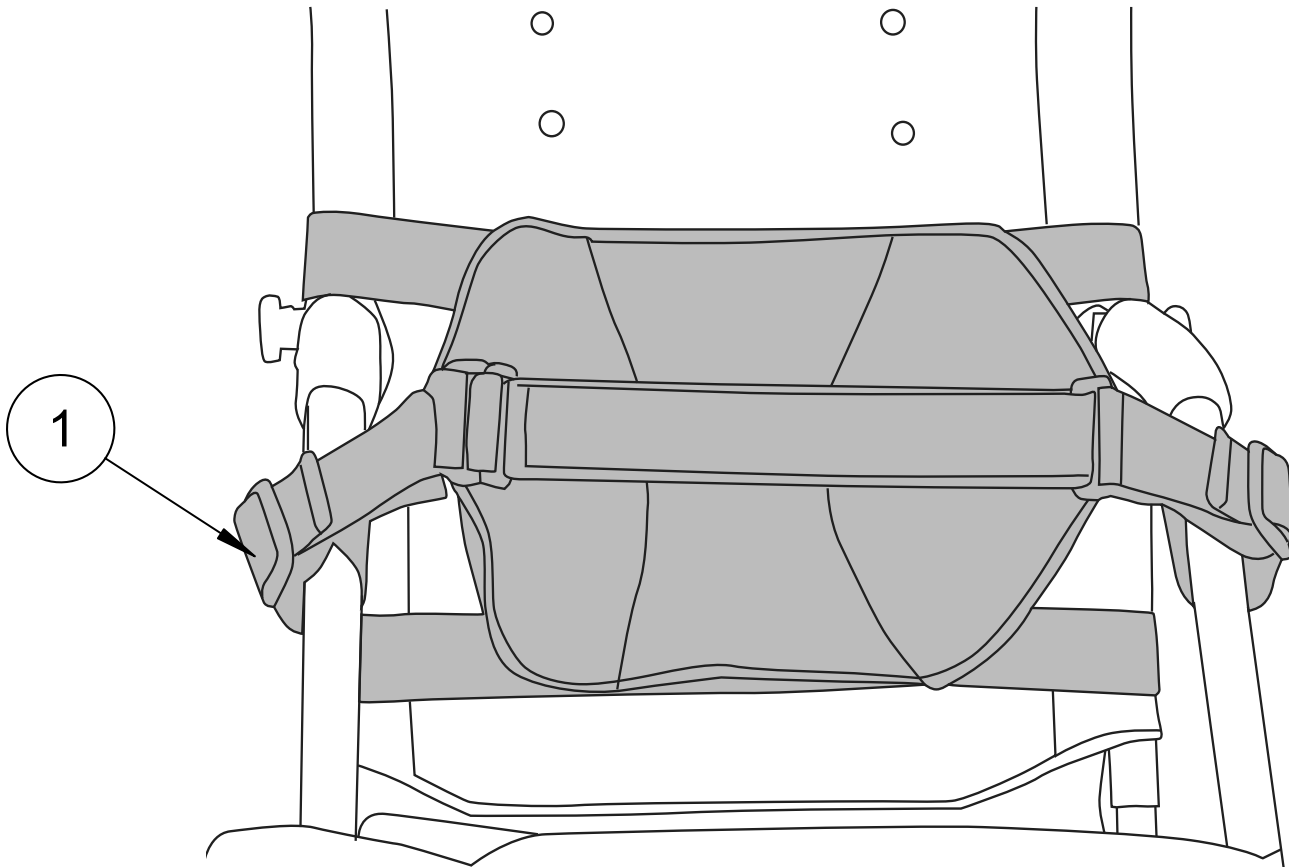




If you have any questions about attaching the Lateral Support, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

 **Convaaid**
Now You're Going Places

INSTRUCTIONS

Soft Adjustable Lateral Support - Single Flap

1. Attach the Lateral Trunk Support to the chair by wrapping the two rear straps around the back of the chair and securing with Velcro®.
 2. Attach the single flap, by wrapping the strap on the left flap around the side tubing of the frame, and securing with Velcro® (repeat steps on opposite side)
 3. Thread the scoliosis strap through the plastic loop on the left flap, back through the plastic loop on the opposite end of the strap, and pull tight.
 4. Bring the scoliosis strap across the chest, thread it through the plastic loop on opposite flap, and fasten with Velcro®.
- To support the right side of the trunk, attach the support to the chair with the double-sided Velcro® strap on the left side. Detach the strap on the right flap from the tubing, and tighten scoliosis strap until desired trunk positioning is obtained.
 - To support the left side of the trunk, attach the support to the chair with the double-sided Velcro® strap on the right side. Detach the strap on the left flap from the tubing, and tighten scoliosis strap until desired trunk positioning is obtained.

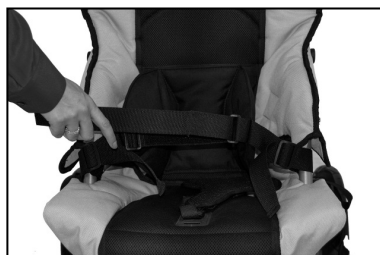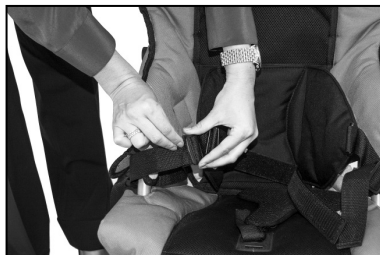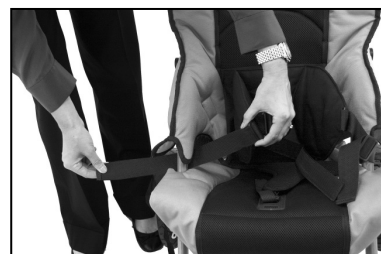

If you have any questions about attaching the Lateral Support, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

 **Convaid**
Now You're Going Places