


ACCESSORIES

Under Seat Storage Basket

995937

Item	Description	Qty
1	Under Seat Storage Basket	1

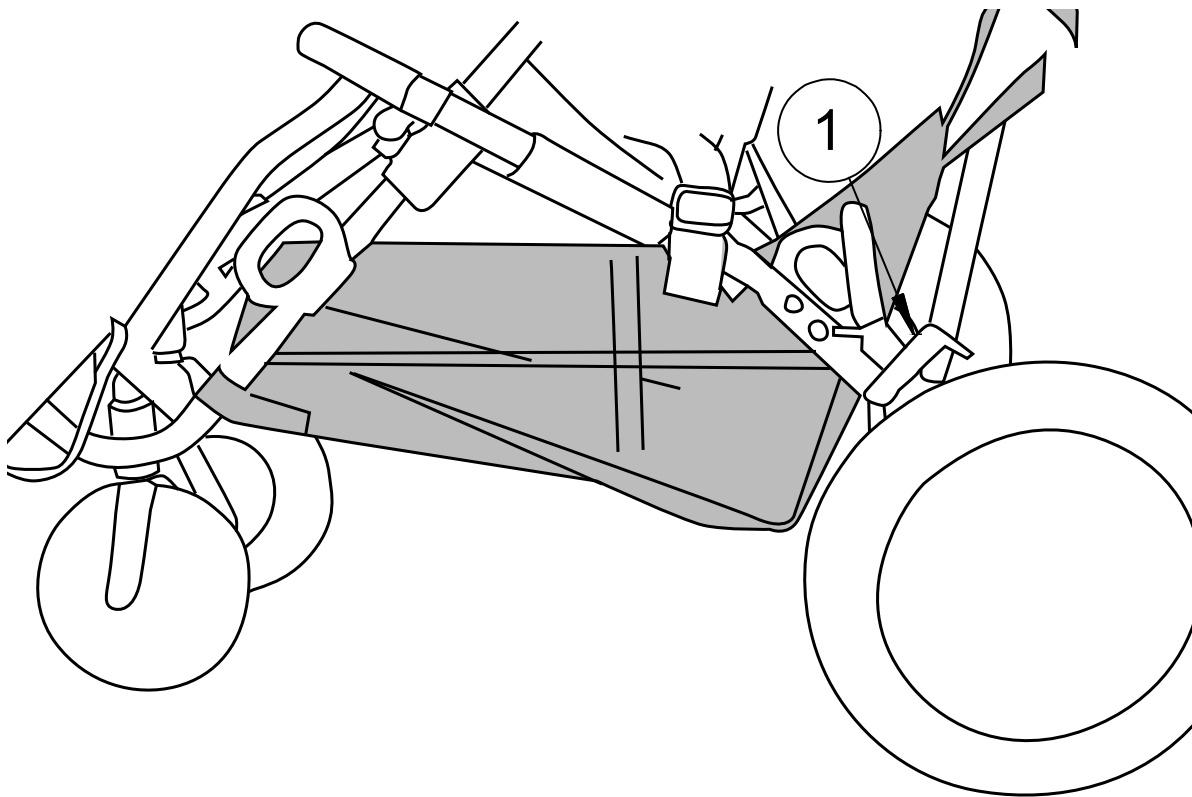

If you have any questions about attaching the Under Seat Storage Basket, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

Under Seat Storage Basket

1. Connect Center Button Strap by looping it over Rear Cross Brace and snapping button to back of basket.
2. Loop Side Button Strap over Chair Tubing (just above the Rear Axle Assembly), and snap button shut. Repeat for opposite side.
3. Loop Front Button Strap over Chair Tubing (just above Front Cross Brace Mount), and snap button shut.