


## TRUNK POSITIONING

### Upper Extremity Support Surface Tray

# 995938

#### Kit Includes

Item	Description	Qty
1	Upper Extremity Support Surface Tray	1
2	Hardware	1

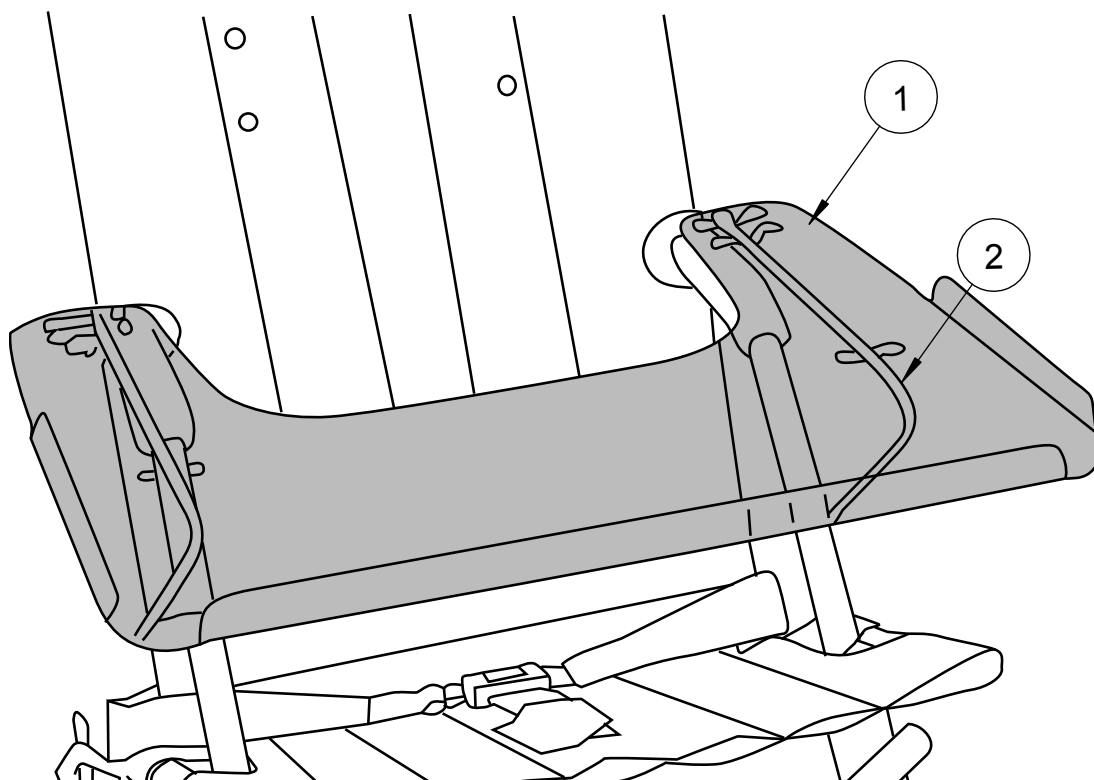

If you have any questions about attaching the Support Tray, please call our Sales Department at 888 CONVAID (266-8243) or +1 310 618-0111.

## **INSTRUCTIONS**

---

### **Upper Extremity Support Surface Tray**

#### **ATTACHING THE TRAY**

1. Raise front of the tray with the tray supports completely unfolded.

**NOTE: For Scout option, raise armrest before engaging knobs.**

2. Engage knobs on side of chair with tray hooks underneath.
3. Lower tray until supports are positioned over the outside of the front legs.