
SET-UP AND ADJUSTMENTS

Unfolding

1. Lay chair flat on the ground and unbuckle closure strap. **Fig. 1**
2. Stand chair on front casters, grasp on to the side of Push Handle and push down on Seat Tube. **Fig. 2**
3. Press down firmly on seat tubes to ensure the chair is completely unfolded. **Fig. 3**
4. Push down on Rear Lock Brace until it locks into a straight position. **Fig. 4**
5. Swing Footplates down into position. **Fig. 5**



Fig. 1



Fig. 2



Fig. 3



Fig. 4



Fig. 5

Folding

1. Apply Wheel Locks. **Fig. 6**
2. Swing Footplates to the side. **Fig. 7**
3. Pull up on the Rear Lock Brace to unlock. **Fig. 8**
4. Grasp on to the Push Handle and simultaneously pull the Seat Tubes up. **Fig. 9**
5. Lay chair flat on the ground with seat facing forward and push down on front leg tubes to complete fold. **Fig. 10**
6. Buckle Closure Strap to keep chair securely closed. **Fig. 11**



Fig. 6



Fig. 7



Fig. 8



Fig. 9



Fig. 10



Fig. 11

LOCKING AND UNLOCKING WHEELS

To lock: Pull down on handle.

To release lock: Pull up on handle.



2



2830 California Street, Torrance, CA
Toll free in the US: 1-888-CONVAID (266-8243) • Phone: (310)

Email: sales@convaid.com • Email: international@convaid.com

www.convaid.com



RECLINE

1. To recline seat back, loosen the Seat Back Knobs. **Fig. 12**
2. Pull seat back and retighten the Back Seat Knobs. **Fig. 13**



Fig. 12



Fig. 13

SEAT DEPTH

To Adjust:

3. Push-in and hold the Grey Locking Pin on both sides of seat. Grasp tubes and push in or pull out to desired length. **Fig. 14a & 14b**



Fig. 14a

Fig. 14b



2830 California Street, Torrance, CA 90503

Toll free in the US: 1-888-CONVAID (266-8243) • Phone: (310) 618-0111 • Fax: (310) 618-8811

Email: sales@convaid.com • Email: international@convaid.com

www.convaid.com

HEADREST EXTENSION

To Attach:

1. Insert Headrest Extension Tube into Seat Back Tubes. **Fig. 15a & 15b**
2. Attach Velcro® to Headrest Extension to each side of the Seat Back. **Fig. 16**



Fig. 15a



Fig. 15b



Fig. 16

OCCI-HEADWING

To Attach:

1. Insert string through grommet hole and tie to secure. Repeat on opposite side. **Fig. 17a & 17b**



Fig. 17a



Fig. 17b

Headrest Cover (Canopy)

To Attach:

1. Remove Canopy from box. **Fig. 18**
2. Align plastic Canopy clips with upper tubing on the back of the chair. **Fig. 19**
3. Push clips until they snap into place. **Fig. 20**
4. Secure the Canopy to the frame by using the Velcro® Straps on the bottom of the Canopy **Fig. 21**



Fig. 18a



Fig. 19b



Fig. 20



Fig. 21

To Open:

1. Grasp top of Canopy and push forward and rotate downward.
Fig. 22a, 23b, & 24c

To Close:

1. Push Canopy back and rotate upward.
Reverse Canopy Opening Steps: **Fig. 22a, 23b, & 24c**



2830 California Street, Torrance, CA 90503

Toll free in the US: 1-888-CONVAID (266-8243) • Phone: (310) 618-0111 • Fax: (310) 618-8811

Email: sales@convaid.com • Email: international@convaid.com

www.convaid.com



Fig. 22a



Fig. 23b



Fig. 24c

TRUNK SUPPORT

To Attach Anterior Vest:

1. Insert harness strap with bolt end through the desired height grommet hole on the seat back upholstery. **Fig. 25**
2. Attach strap to Seat Back by tightly securing knob to the harness bolt on the seat back upholstery. **Fig. 26**
3. Fasten buckle and adjust straps to secure the user. **Fig. 27**

To Attach H-Harness:

Note: See instructions 1-2 listed above before

3. Attach each H-Harness clip to the metal clips on the Lap Belt and clip H-Harness buckle together. **Fig. 28a & 28b**



Fig. 25



Fig. 26



Fig. 27



2830 California Street, Torrance, CA 90503

Toll free in the US: 1-888-CONVAID (266-8243) • Phone: (310) 618-0111 • Fax: (310) 618-8811

Email: sales@convaid.com • Email: international@convaid.com

www.convaid.com



Fig. 28a



Fig. 28b

PELVIC SUPPORT

Three Point Positioning Belt

To Buckle: Insert metal clasp into the sides of the buckle. **Fig. 29**

To Release: Press the grey button on the buckle and pull out the clasps. **Fig. 30**



Fig. 29



Fig. 30

Height Adjustable Flip-up Armrest

WARNING: APPLY WHEEL LOCKS BEFORE ATTACHING ARMREST

To Attach:

1. Insert Armrest tube to Armrest attaching bracket. **Fig. 31**
2. Adjust Armrest height by pushing the Grey Locking Pin and release Grey Locking Pin to desired height. **Fig. 32**
3. Secure Armrest into attaching bracket with Grey Locking Pin. **Fig. 33**



Fig. 31

Fig. 32

Fig. 33

GRAB BAR

To Attach (Requires Height Adjustable Armrest):

1. Remove Armrest cap cover. **Fig. 34**
2. Align the Grab bar tubes with the Armrest tubes and push the Grey Locking Pins in, while simultaneously pushing Grab Bar into the Armrest tubes. **Fig. 35**
3. Push Grey Locking Pins into Armrest until the locking pins lock into place. **Fig. 36**
4. Grab Bar Attached to Armrest. **Fig. 37**



Fig. 34



Fig. 35

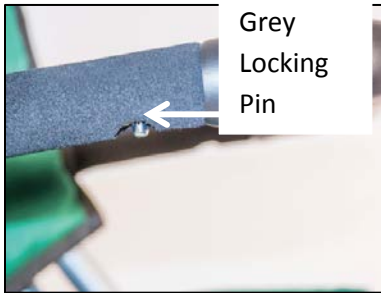


Fig. 36



Fig. 37

Upper Extremity Support Surface Tray (Padded Tray)

To Attach (Requires Height Adjustable Armrest and Mounting Tray Hardware):

1. Unfasten the Velcro® located on the bottom of the padded tray. **Fig. 38**

10



2830 California Street, Torrance, CA 90503

Toll free in the US: 1-888-CONVAID (266-8243) • Phone: (310) 618-0111 • Fax: (310) 618-8811

Email: sales@convaid.com • Email: international@convaid.com

www.convaid.com

2. Align the Tray Mounting Bar with the Armrest tubes and push the Grey Locking Pins in, while simultaneously pushing Tray into the Armrest tubes. **Fig. 39**
3. Push Grey Locking Pins into Armrest until the locking pins lock into place. **Fig. 40**
4. Secure the Tray by wrapping the Velcro® strap around the Armrest. **Fig. 41**



Fig. 38



Fig. 39



Fig. 40



Fig. 41

UPHOLSTER

Seat Upholstery Installation:

1. Place seat flat onto the mesh retaining strap. **Fig. 42**
2. Thread Velcro® strap through eyelet and fasten. **Fig. 43a & 43b**
3. Using a Phillips screw driver attach strap into the seat depth hole, based on seat depth adjustment required, on both sides of the seat tube. **Fig. 44**
4. Tightly wrap the bottom Velcro® straps around the bottom of the seat. **Fig. 45**

Note: To remove seat upholstery repeat instructions in reverse order.



Fig. 42



Fig. 43a



Fig. 43b



Fig. 44



Fig. 45

Upholstery Installation:

1. To attach and adjust the Tension Back, fasten all Velcro® straps around the back of the frame, starting from the top and fasten Velcro Side Straps on each side. **Fig. 47a & 47b**

Fig. 47a & 47b

Note: To remove back upholstery repeat instructions in reverse order.



Fig. 47a



Fig. 47b

FOOT & LEG POSITIONING

Foot Positioners

Criss-cross straps over foot and fasten Velcro® **Fig. 48**



Footplate Securement Strap

To secure floorplates together and prevent them from flipping up and down, buckle the left and right side together. **Fig. 49a & 49b**



Fig. 49a



Fig. 49b

Footplate Depth Adjustment

Using a hex wrench to loosen the two bolts located on the top of the footplate and slide footplate forward or rearward to desired depth and retighten bolts. **Fig. 50**



Fig. 50